

INFORMATION BULLETIN

May 2026

CHANGE OF ADHESIVE GLUE FOR PERFORMA CG AIR CUSHION REPAIRS

INTRODUCTION

As part of the continuous improvement of our repair kit offering, the 15-patch repair kit (PCGAPK15) will be discontinued and only one repair kit format will be available: **PCGAPK3**.

PREVIOUS FORMAT VS NEW FORMAT

The PCGAPK15 repair kit will be removed from our service product offering. The PCGAPK3 kit will now be the only repair kit offered.

The PCGAPK3 kit continues to include all essential components required to perform repairs efficiently. The 20g tube is designed to improve adhesive preservation after opening, while still providing more than enough quantity to complete the 15 repairs included in the kit.

SHELF LIFE AND STORAGE CONDITIONS

The adhesive has a shelf life of 18 months from the date of manufacture.

The following storage conditions are recommended:

- Store in a cool, dry place, at a temperature above freezing.
- Avoid direct exposure to sunlight.
- Close the tube immediately after each use.
- Avoid exposure to high humidity, as it may reduce shelf life.
- Avoid exposure to high temperatures (> 50 °C), as they may alter the product (e.g., yellowing of the adhesive).

OTHER CONSIDERATIONS

Pricing and Order Forms

- The PCGAPK15 kit will no longer be available for order.
- All order forms have been updated accordingly and now reflect the exclusive availability of the PCGAPK3 kit.

PERFORMA CG AIR GLUE (cont'd)**Resources**

Documents:

All pertinent documents have been updated and are online.

- [Website](#)
- [Order Forms USA](#)
- [Order Forms CA](#)
- [Order Forms ON](#)
- [Parts Manual](#)

Videos:

Repair videos were created and are online under *Support / For Pros / [Instruction Guides - Videos](#)*.

- Repairing a tear at the base of the cell
- Repairing a detached valve
- Repairing a puncture from a sharp object