

Bluetooth Pairing Quick Start Guide

R-Net Advanced Joystick and OMNI 2



Steps for pairing either the Advance Joystick or the OMNI 2 with any devices (Android, Apple or PC) :

1 - TURN ON THE BLUETOOTH MODE ON THE JOYSTICK/OMNI 2:

- In the settings menu, select Bluetooth. (To enter the settings menu, press and hold top button: on the left of the screen for the joystick; and on the right of the screen for the OMNI 2).
- Set target device(s) to < On >. ("Mouse" for Android or PC and "iDevice" for Apple products)
- The joystick/OMNI 2 must be switched off and on again.



2 - SET THE JOYSTICK OR OMNI 2 IN DISCOVERY MODE:

- On the joystick/OMNI 2, navigate to the Bluetooth mode by pressing the < mode > button.
- For the joystick: Toggle forward and hold until a beep is heard (about 10 seconds). Then, toggle backward and hold until a beep is heard (about 10 seconds).
- For the OMNI 2: Depending on the chair's specialty driver control, activate the control used to move the chair forward and hold until a beep is heard (about 10 seconds), then release. Next, activate the control used to move the chair backward and hold until a beep is heard (about 10 seconds), then release.
- The Bluetooth symbol should blink on the top of the screen. This means that the joystick/OMNI 2 is in discovery mode.



3 - THE DEVICE CAN NOW BE PAIRED WITH THE JOYSTICK OR THE OMNI 2...

For more information on how to pair a device with either the joystick or the OMNI 2, or on the Bluetooth functionality, please refer to the Curtiss-Wright document (Chapter 6) on our website at www.amylior.com.



Phone : +1 450 424-0288 | Fax : +1 450 424-7211
info@amylior.com | amylior.com