


A1	106835	SEAT UNIT BUG Medium LIGHT GREY GREEN UPHOLSTERY
A2	106836	SEAT UNIT BUG Medium DARK GREY GREEN UPHOLSTERY
A3	106837	SEAT UNIT BUG Medium LIGHT GREY PINK UPHOLSTERY
A4	106838	SEAT UNIT BUG Medium DARK GREY PINK UPHOLSTERY
B	106830	869 Medium FOUR WHEEL BASE FOR BUG BLACK FRAME
D	108118	856 Medium HIGH LOW ADJUST BASE BUG

POS. PART NR. DESCRIPTION

### Spare Parts BUG size Medium

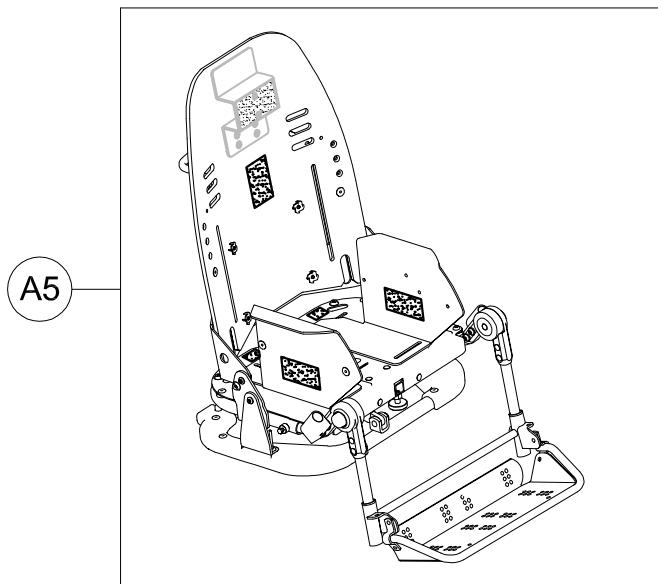

A5



GRIGIO CHIARO VERDE  
LIGHT GREY GREEN

A41



GRIGIO CHIARO ROSA  
LIGHT GREY PINK

A42




GRIGIO SCURO VERDE  
DARK GREY GREEN

A43





GRIGIO SCURO ROSA  
DARK GREY PINK

A44

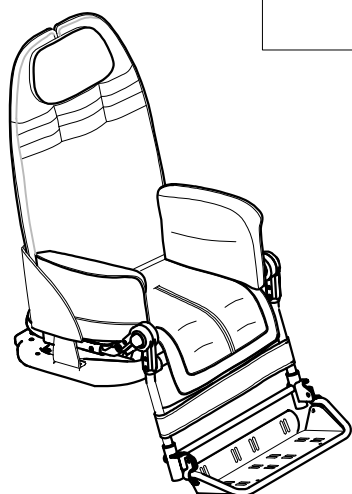

A5	105602	BUG Medium SEAT UNIT (NO UPHOLSTERY)
A41	171645	LIGHT GREY GREEN UPHOLSTERY BUG Medium
A42	171646	LIGHT GREY PINK UPHOLSTERY BUG Medium
A43	171647	DARK GREY GREEN UPHOLSTERY BUG Medium
A44	171648	DARK GREY PINK UPHOLSTERY BUG Medium
POS.	PART NR.	DESCRIPTION








### Spare Parts BUG Medium SEAT UNIT

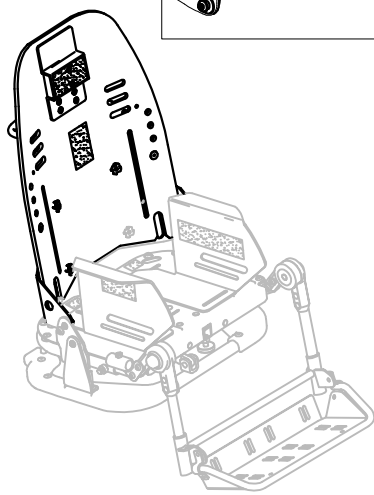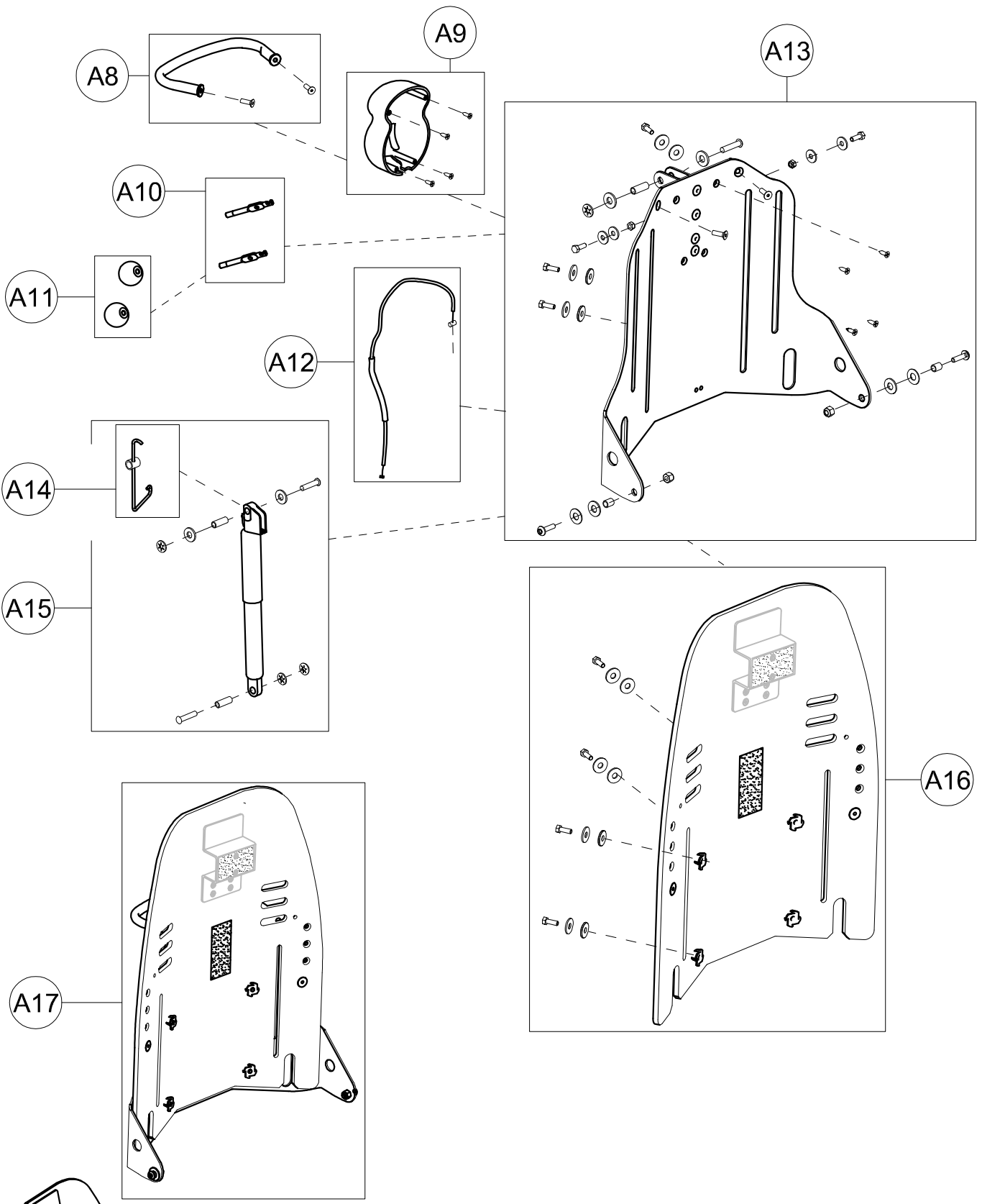

A8	301355	COMPLETE HANDLE OF THE SEATING UNIT FOR BUG
A9	301356	COMPLETE COVER OF THE BACKREST FOR BUG
A10	301965	PAIR OF INCLINATION/TILTING CONTROL LEVERS BUG
A11	301966	PAIR OF KNOB D30 RED / BLACK BUG
A12	21351	CABLE + COMPLETE SHEATH OF THE SEAT FOR BUG Medium
A13	301362	COMPLETE ADJUSTMENT OF THE BACKREST FOR BUG Medium
A14	301969	SPRING, Ø0, 7 + BUSHING FOR GAS SPRING
A15	301968	COMPLETE GAS SPRINGS BACKREST FOR BUG Medium
A16	303349	BACKREST + SCREWS FOR BUG Medium
A17	303351	COMPLETE BACKREST BUG Medium
POS.	PART NR.	DESCRIPTION

Spare Parts BACKREST STRUCTURE mis. Medium



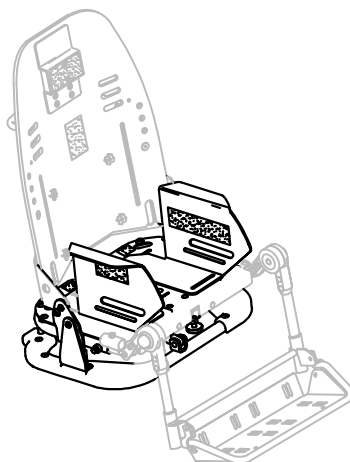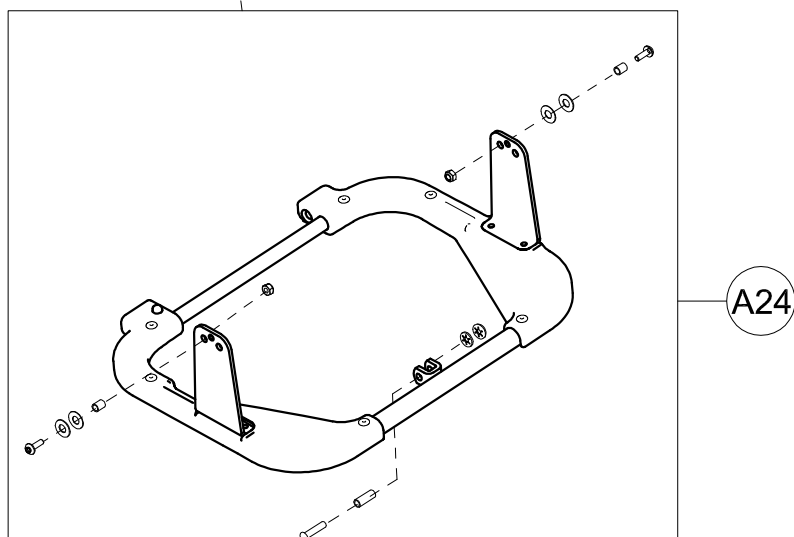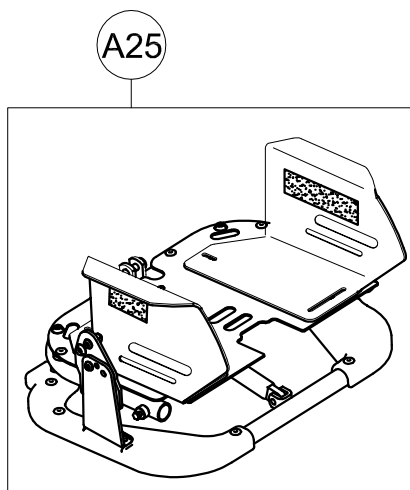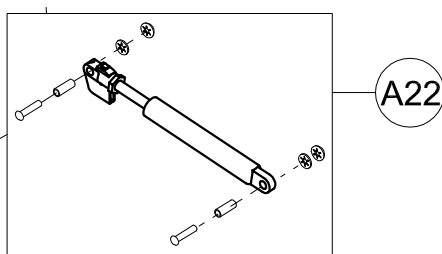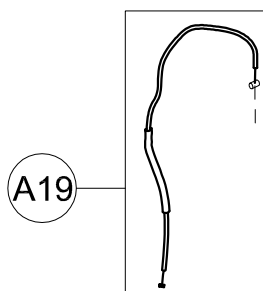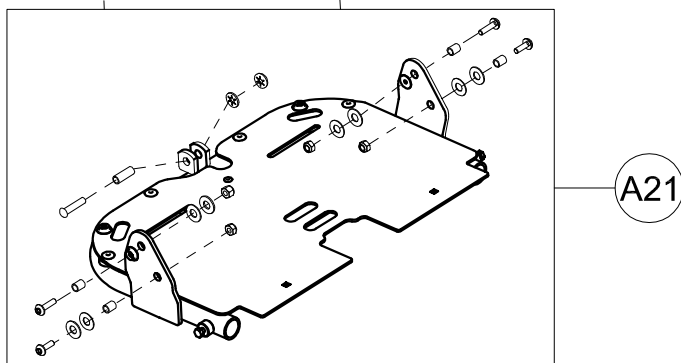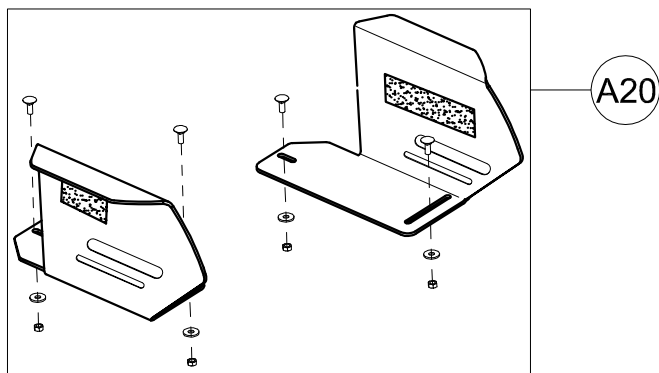

A19	21351	CABLE+ SHEATH SEAT BUG Medium
A20	302044	COMPLETE RIGHT / LEFT SIDES OF THE SEAT FOR BUG Medium
A21	301349	COMPLETE SEAT FOR BUG Medium
A22	301587	GAS SPRING FOR BUG TILT + SCREWS
A24	301352	LOWER SEAT FOR BUG Medium
A25	301448	COMPLETE LOWER SEATING FRAME BUG Medium
POS.	PART NR.	DESCRIPTION


Spare Parts LOWER SEATING FRAME Size Medium

pag. 4


**ORMESA**

**BUG**



THIS DRAWING BELONGS TO ORMESA: IT IS FORBIDDEN TO REPRODUCE, CHANGE OR INFORM ANY THIRD PARTY ABOUT ITS CONTENTS

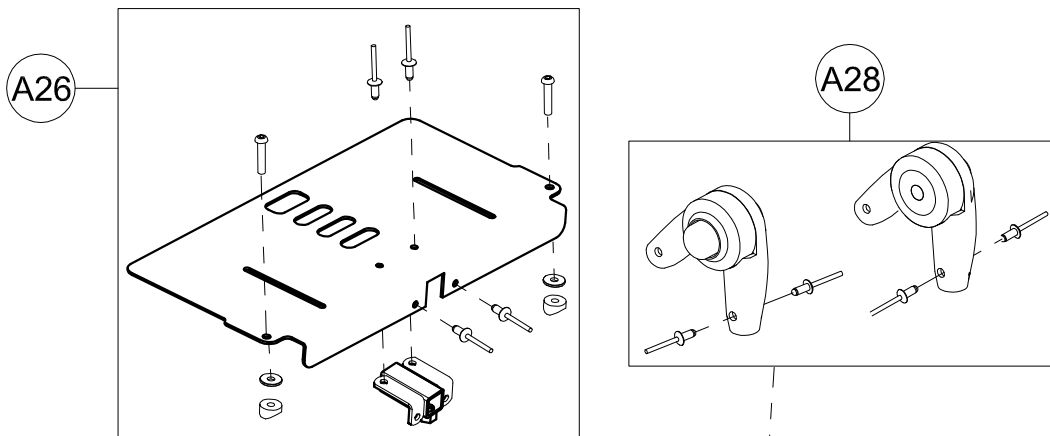

A28

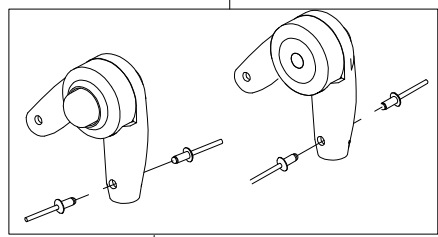


A27

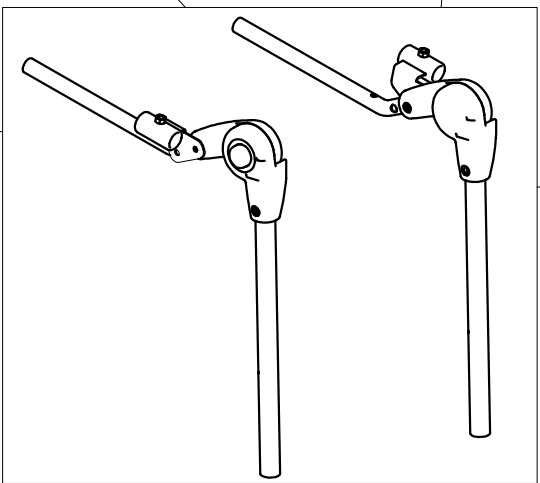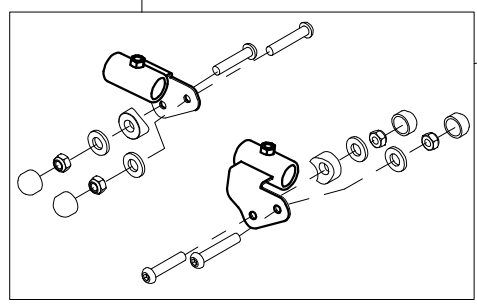


A29

A33


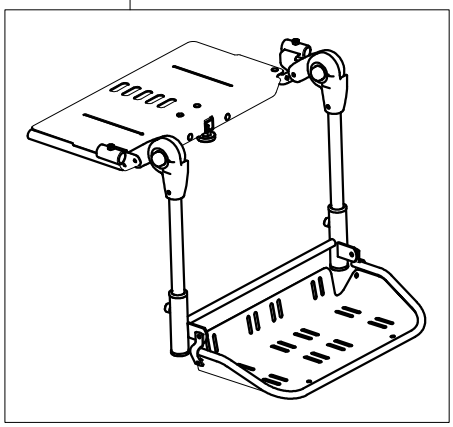


A30

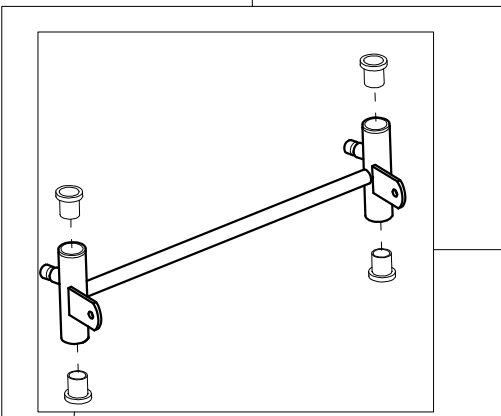

A31

A32

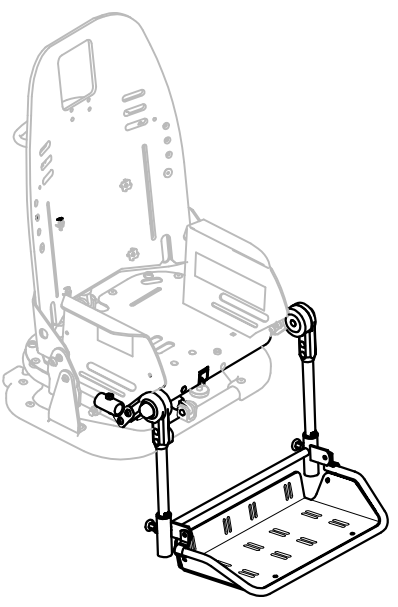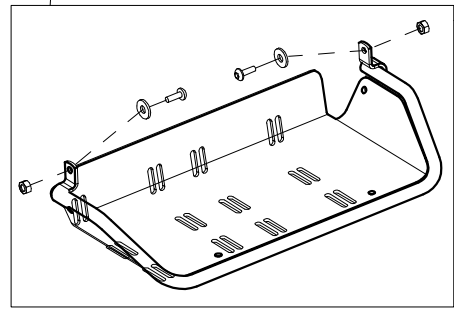


A26	301973	EXTENDED SEAT BUG Medium +JOIN PIECE OF THE ABDUCTION BLOCK
A27	303063	PAIR OF TRAY ATTACHMENT BUG
A28	301974	PAIR OF LEGREST PVC JOINTS FOR BUG Medium
A29	303263	RIGHT/ LEFT LEGRESTS + JOINTS BUG Medium
A30	301601	COMPLETE FOOTREST FOR BUG Medium 3R-4R
A31	301602	COMPLETE TROUSER HEM-BINDING FOR BUG Medium 3R/4R
A32	301440	COMPLETE FOOTREST BUG Medium WITHOUT HEM-BINDING
A33	303265	FOOT AND LEGREST WITH EXTENDED SEAT BUG Medium
POS.	PART NR.	DESCRIPTION

Spare Parts

FOOT AND LEGREST WITH EXTENDED SEAT size Medium



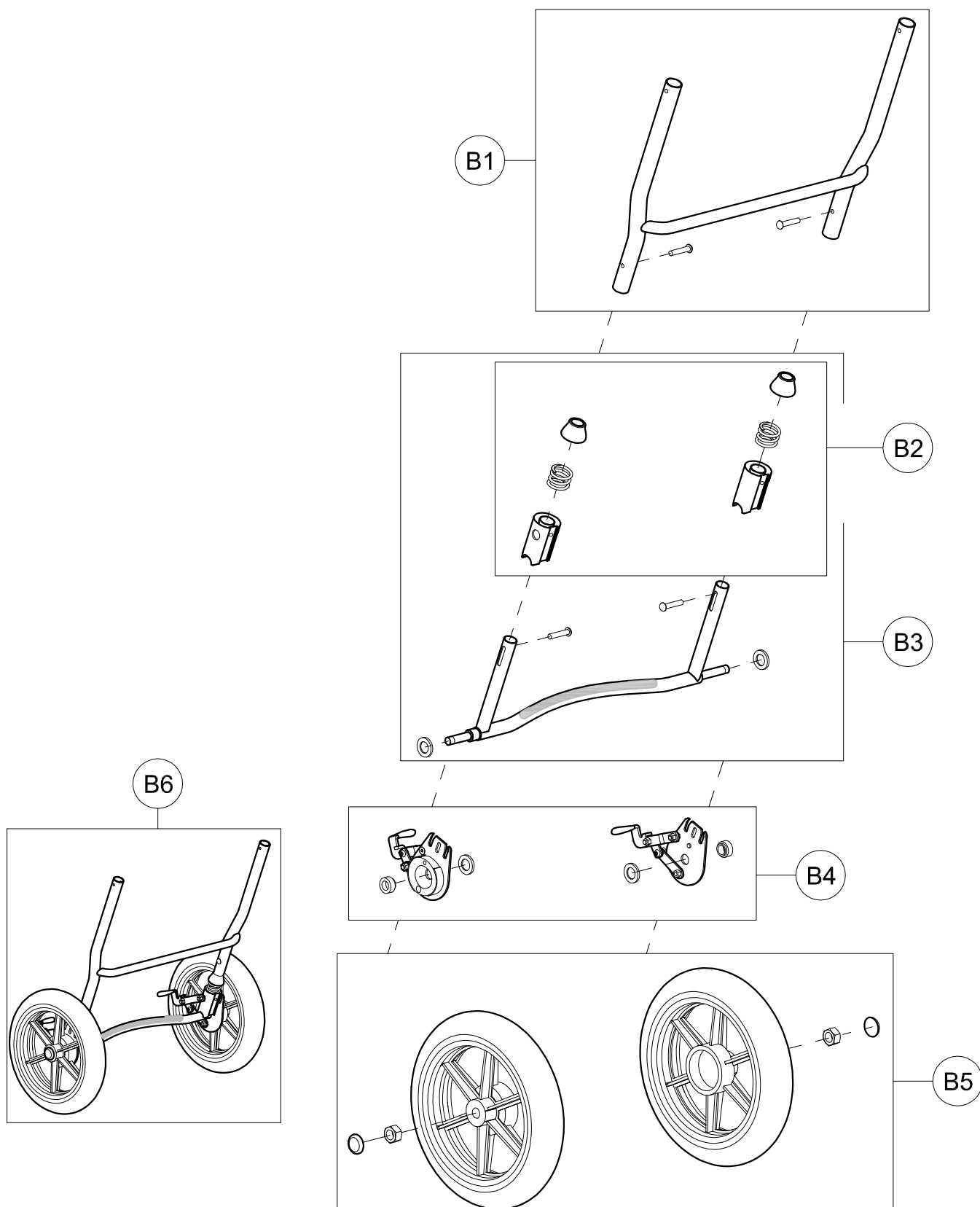


B1	301976N	BACK FRAME FOR BUG Medium 4R (BLACK)
B2	301980	SET OF SHOCK ABSORBERS FOR BUG Medium
B3	301978N	COMPLETE FRAME OF SHOCK ABSORBER FOR BUG Medium 4R
B4	301981	PAIR OF RIGHT-LEFT BRAKE COMPLETE BUG
B5	301983	PAIR OF REAR WHEELS 12" BUG Medium
B6	301985N	COMPLETE BACK FRAME OF THE 869 BASE BUG Medium
POS.	PART NR.	DESCRIPTION


Spare Parts BACK FRAME OF THE 869 BASE size Medium

**ORMESA**


**BUG** pag. 6

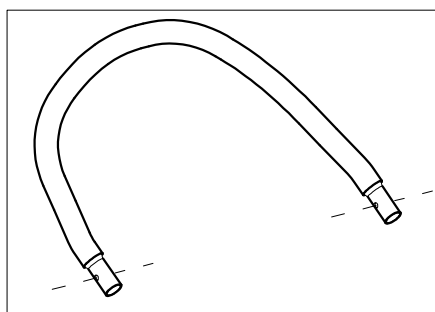


B7

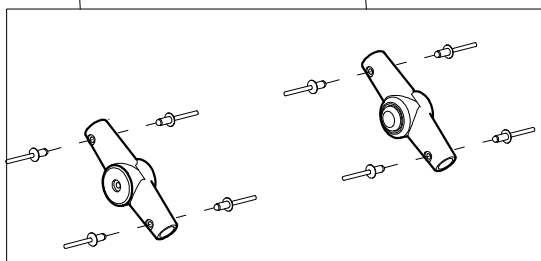


B8

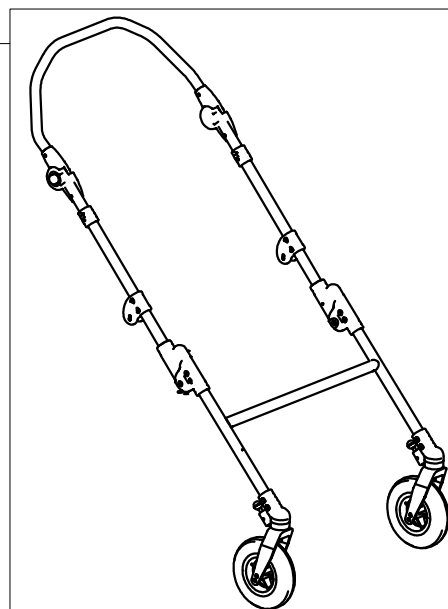


B20

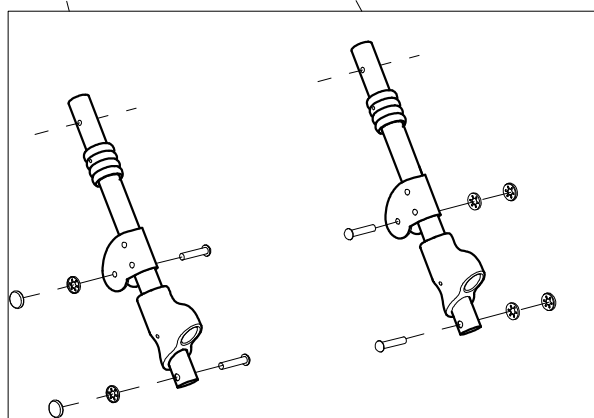


B9

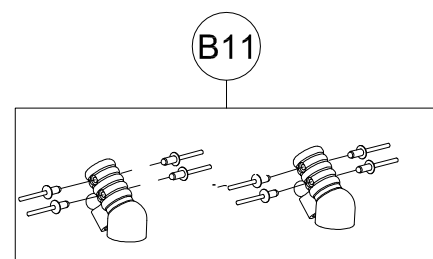


B11

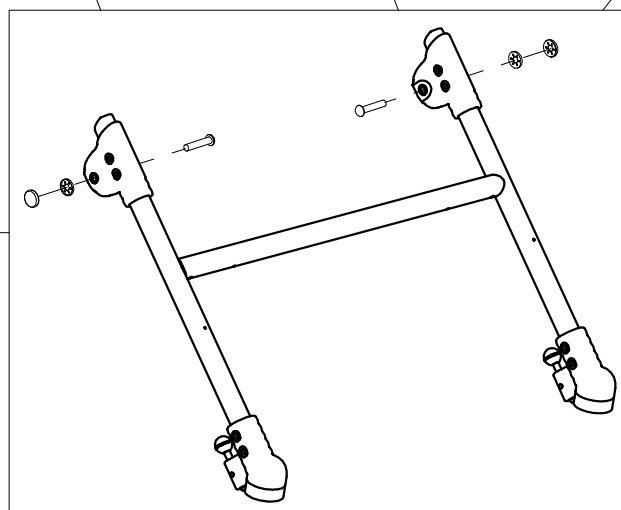


B10

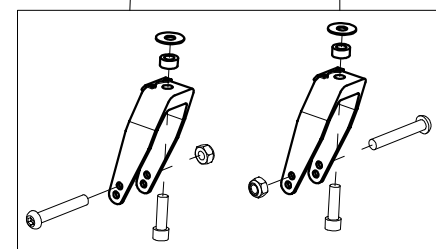

B12

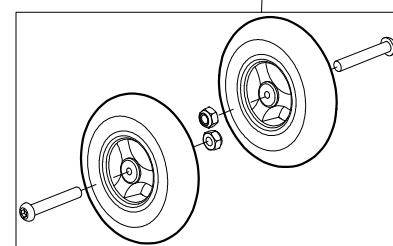

B13





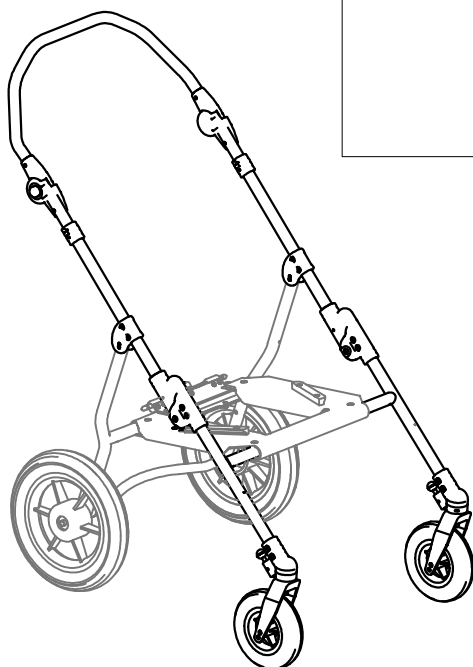


B7	303352	COMPLETE PUSH HANDLE OF BUG Medium
B8	302431	PAIR OF PUSH HANDLE JOINTS WITH RIVET BUG 3R/4R
B9	303353N	PAIR OF THE PUSH HANDLE TUBES FOR BUG Medium 4R
B10	303354N	COMPLETE FRONT FRAME + WHEEL SUPPORT BUG Medium 4R (BLACK)
B11	301991	PAIR OF FRONT WHEEL SUPPORTS FOR BUG 4R
B12	303355	PAIR OF FRONT WHEEL FORKS FOR BUG 4R
B13	303356	PAIR OF FRONT WHEELS FOR BUG 4R
B20	303357N	COMPLETE FRONT FRAME OF THE 869 BASE BUG Medium (BLACK)
POS.	PART NR.	DESCRIPTION

Spare Parts FRONT FRAME OF THE 869 BASE size Medium

**ORMESA**

**BUG**

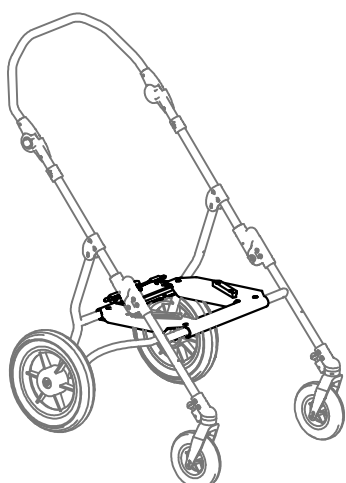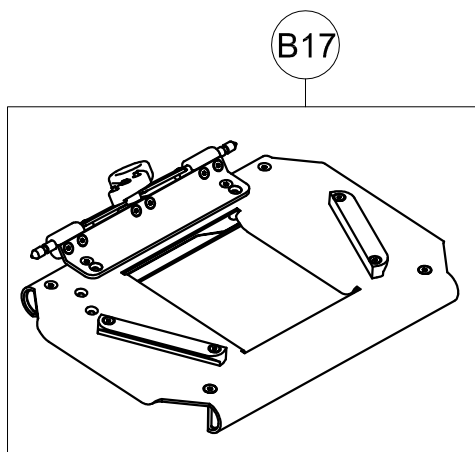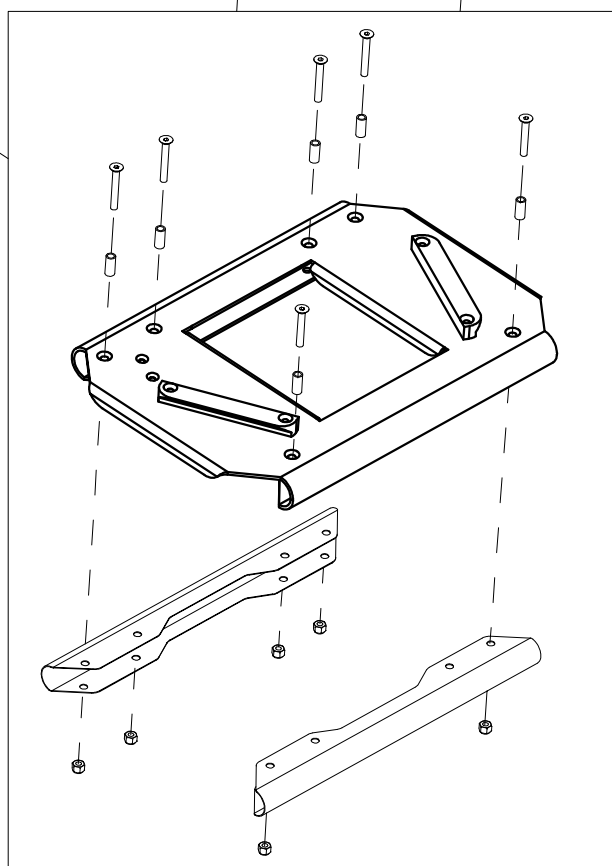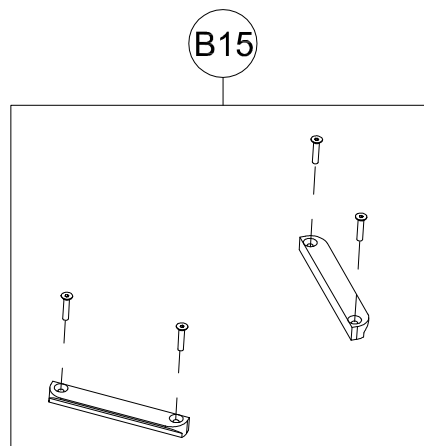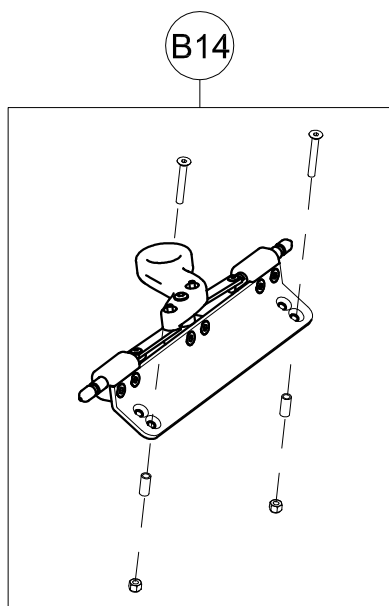


B14	301367	FIXING PART FOR SEAT OF BUG
B15	301996	PAIR OF SEAT SLIDES FOR BUG Medium
B16	302463	SHEET-METAL OF THE SEAT FOR 869 BUG Medium +SLIDES + SCREWS
B17	302469	SEAT OF THE 869 BASE BUG Medium
POS.	PART NR.	DESCRIPTION


Spare Parts SEAT OF THE 869 BASE size Medium pag. 8

**ORMESA**

**BUG**

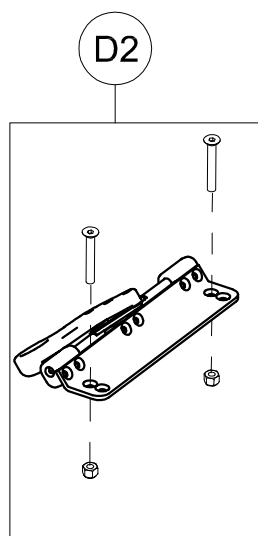


D2

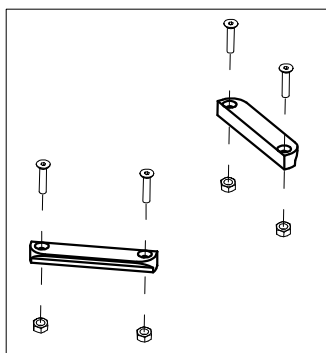

D3


D1

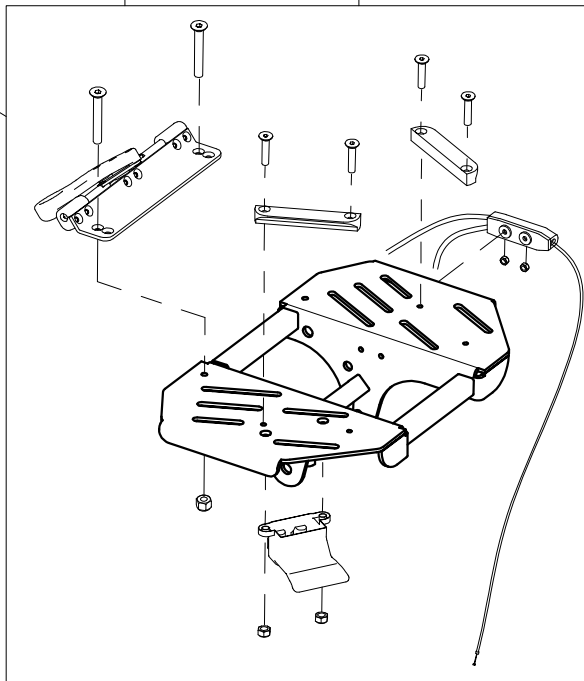


D4





D5

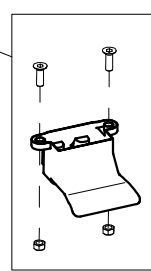

D7

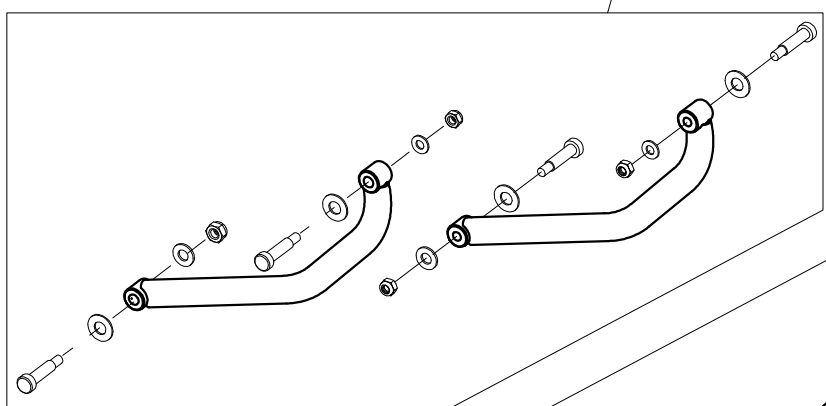


D6

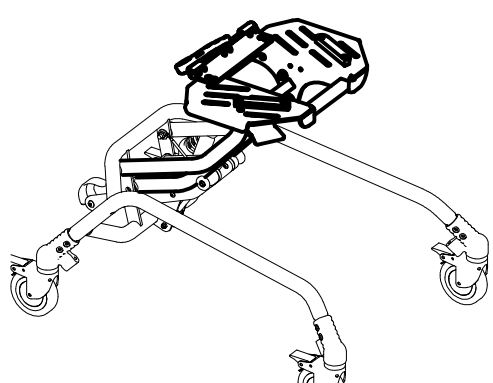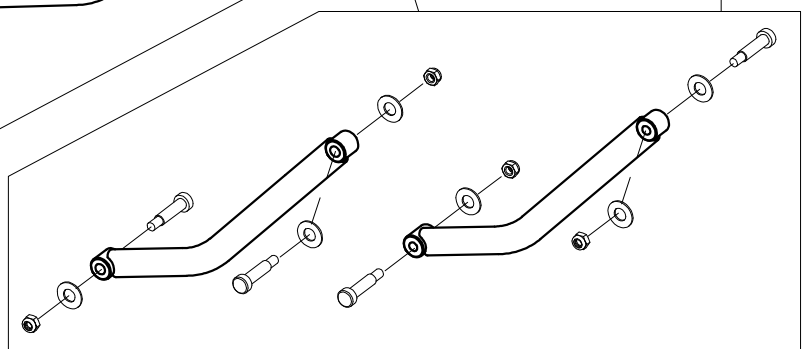

D1	302338	COMPLETE SEAT 856 BUG Medium
D2	301367	FIXING PART FOR SEAT OF BUG
D3	302427	PAIR OF THE SEAT SLIDES FOR BUG Medium
D4	302310	SET OF CABLES+DOUBLER 856 Medium BUG
D5	302335	856-BUG MANUAL LIFTING LEVER + SCREWS
D6	302337	PAIR OF UPPER ARMS OF 856 BASE FOR BUG
D7	302336	PAIR OF LOWER ARMS OF 856 BASE FOR BUG
POS.	PART NR.	DESCRIPTION

**Spare Parts UPPER PART OF THE BASE FRAME 856 size Medium**



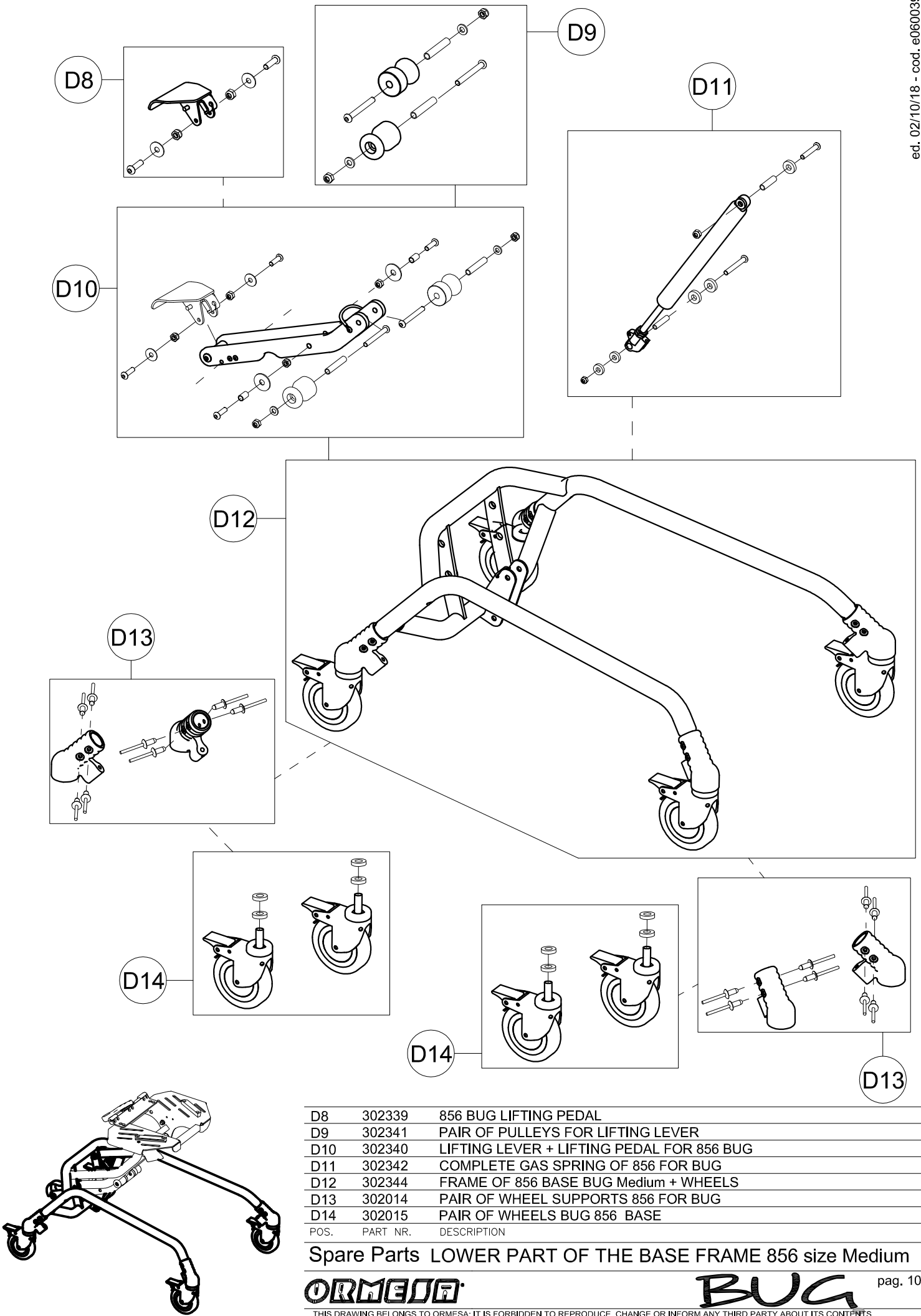

D8	302339	856 BUG LIFTING PEDAL
D9	302341	PAIR OF PULLEYS FOR LIFTING LEVER
D10	302340	LIFTING LEVER + LIFTING PEDAL FOR 856 BUG
D11	302342	COMPLETE GAS SPRING OF 856 FOR BUG
D12	302344	FRAME OF 856 BASE BUG Medium + WHEELS
D13	302014	PAIR OF WHEEL SUPPORTS 856 FOR BUG
D14	302015	PAIR OF WHEELS BUG 856 BASE
POS.	PART NR.	DESCRIPTION

Spare Parts LOWER PART OF THE BASE FRAME 856 size Medium

**ORMESA**

**BUG**