
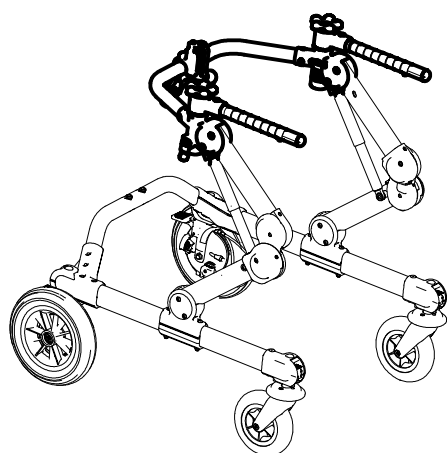



A	103639	925-Small TRUNK SUPPORT
B	103635	924-Small PELVIC SUPPORT
C	103631	930-Small PELVIC HARNESS
D	103573	809-Small ARM SUPPORT
G	102808	890-C Small LEG DIVIDER+ABDUCTOR
H	302450	RIGHT-LEFT SUPPORTS FOR 809
J	303217	FRAME OF POSTERIOR GRILLO small
K	102778	918 Small FOLDING SEAT
L	102792	919 Small LUMBAR SUPPORT
M	103588	927 REMOVABLE GRIP HANDLES
P	103601	815 Small SADDLE
Q	103581	923 PUSH HANDLEBAR FOR CARE GIVERS
R	105604	810 Small PAIR OF WEIGHT BARS
S	102811	890-SC Small LEG DIVIDER
T	102970	939 ERGONOMIC HANDLES
V	103615	943-Small ARM STRAPS
POS.	PART NR.	DESCRIPTION

Spare Parts POSTERIOR GRILLO size Small

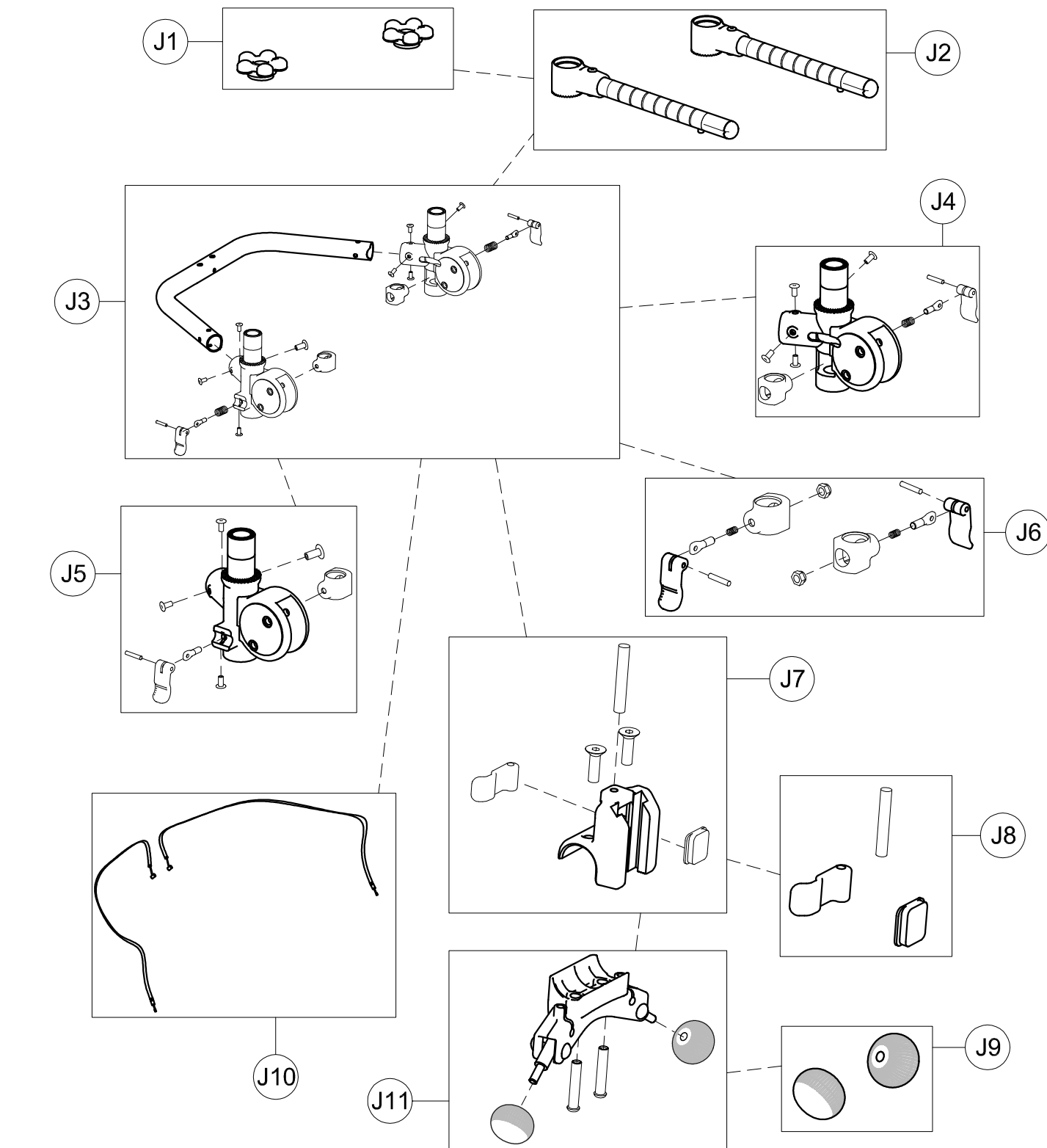



J1	301797	PAIR OF KNOBS TO FIX HORIZONT.SUPPORT
J2	302404	PAIR OF HORIZONTAL BARS
J3	303204	UPPER CROSSPIECE SUPPORT+LOCKING LEVERS
J4	302286	RIGHT UPPER CROSSPIECE SUPPORT+LOCKING LEVER
J5	302287	LEFT UPPER CROSSPIECE SUPPORT+LOCKING LEVER
J6	302288	SET OF LEVERS TO LOCK TRUNK SUPPORT VERTICALLY
J7	302268	UPPER PELVIC BLOCK FIXING
J8	301806	SET OF LEVER TO LOCK PELVIC SUPPORT VERTICALLY
J9	301808	PAIR OF KNOBS FOR GAS SPRING
J10	302407	PAIR OF HEIGHT ADJUST.CABLES
J11	302823	OPERATING LEVER SUPPORT
POS.	PART NR.	DESCRIPTION

**Spare Parts** UPPER PART OF POSTERIOR GRILLO BASE size Small

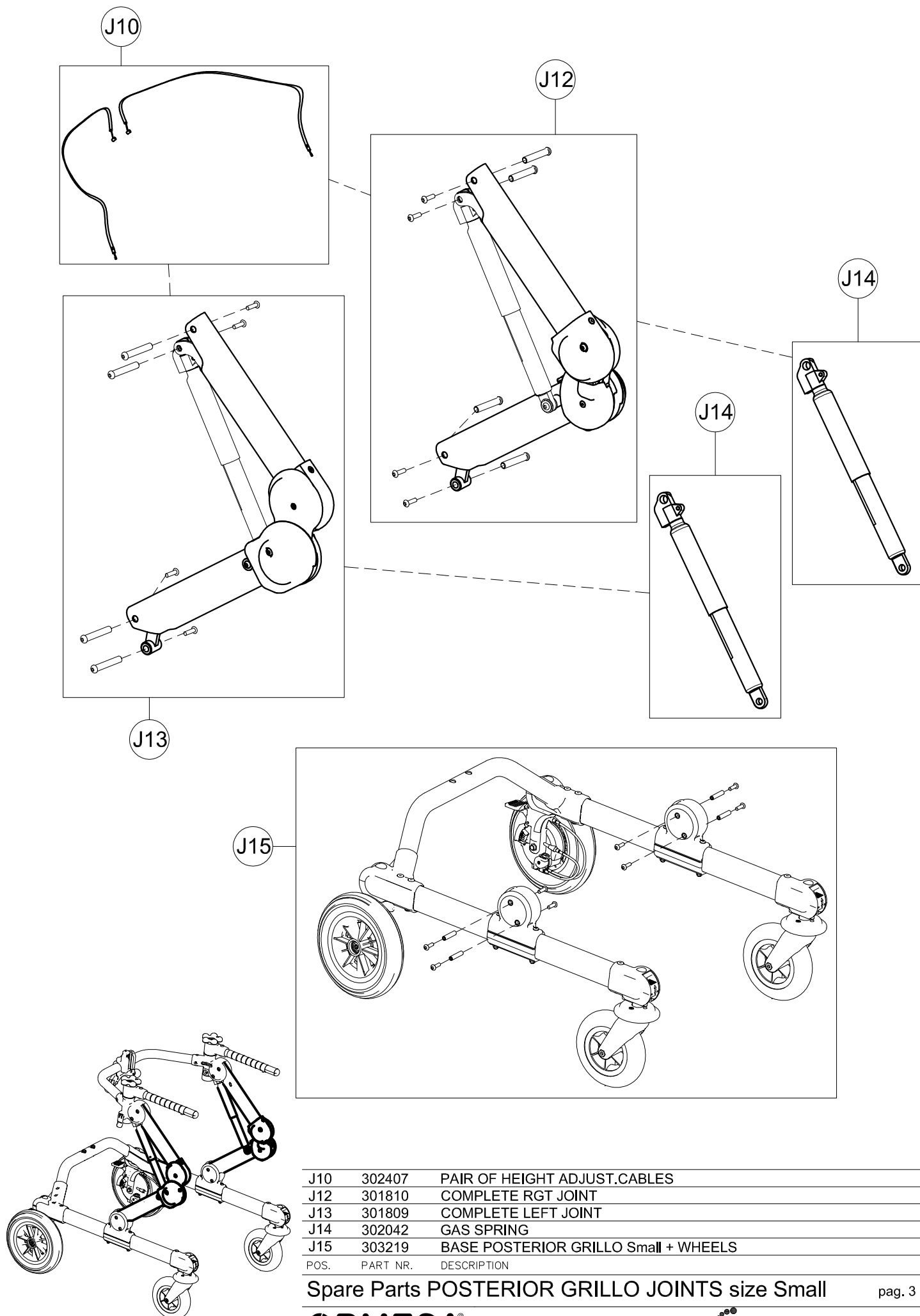



J10	302407	PAIR OF HEIGHT ADJUST.CABLES
J12	301810	COMPLETE RGT JOINT
J13	301809	COMPLETE LEFT JOINT
J14	302042	GAS SPRING
J15	303219	BASE POSTERIOR GRILLO Small + WHEELS
POS.	PART NR.	DESCRIPTION

Spare Parts POSTERIOR GRILLO JOINTS size Small

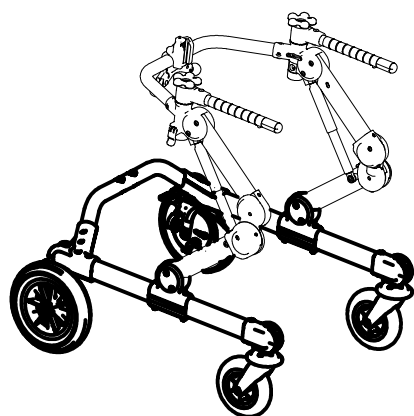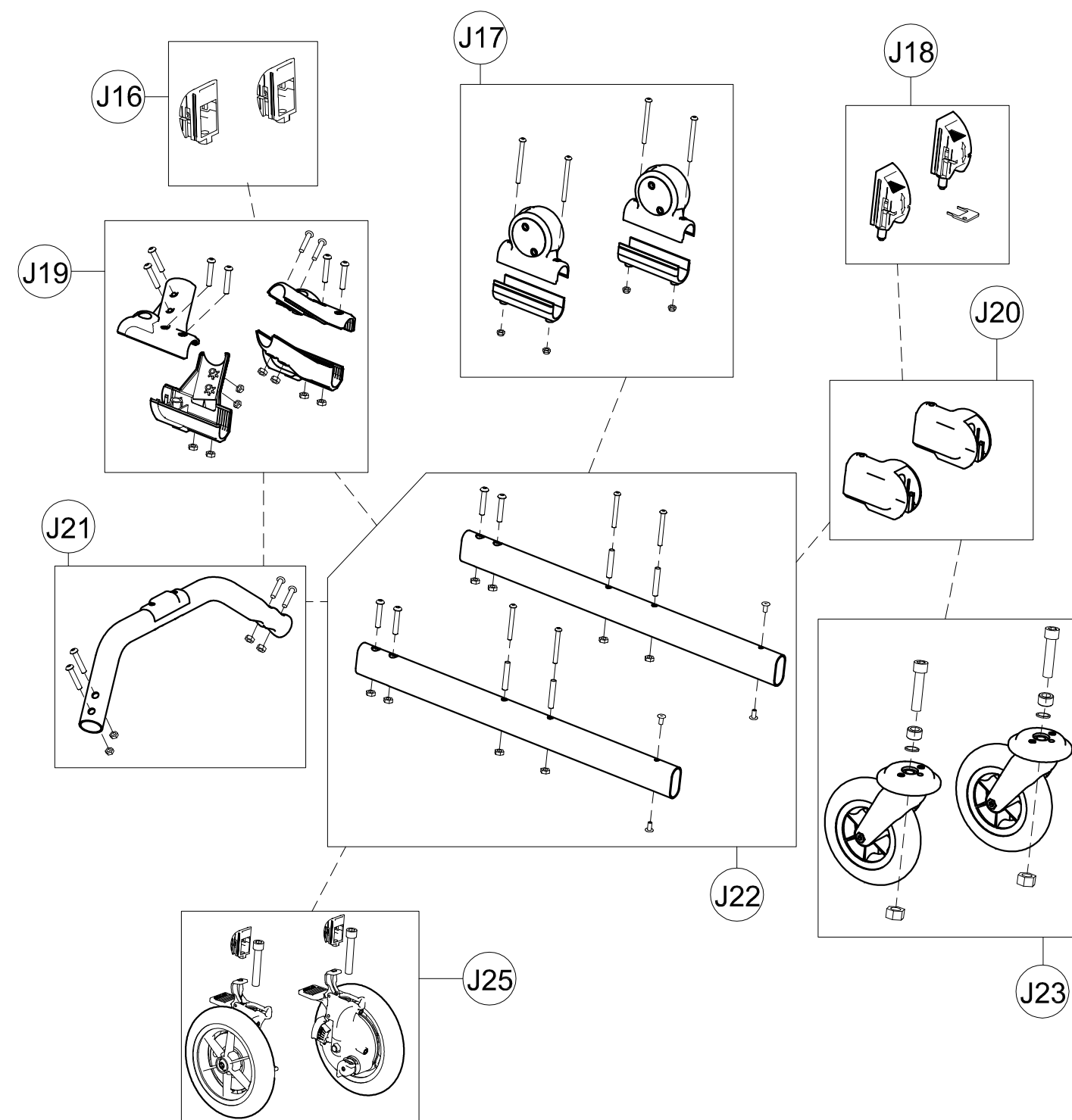

J16	303508	PAIR OF DIRECT.CAPS FOR WHEELS
J17	301812	PAIR OF UPPER/LOWER SUPPORTS OF SIDE
J18	302546	PAIR OF DIRECTION LOCKS
J19	303211	KIT OF LOWER CROSSPIECE SUPPORT
J20	303208	PAIR OF WHEEL SUPPORTS
J21	303213	LOWER CROSSPIECE+SCREWS
J22	303209	PAIR OF BASE SIDES
J23	303385	PAIR OF COMPLETE FRONT WHEELS
J25	303475	PAIR OF COMPLETE REAR WHEELS
POS.	PART NR.	DESCRIPTION



Spare Parts LOWER PART OF POSTERIOR  
GRILLO BASE size Small

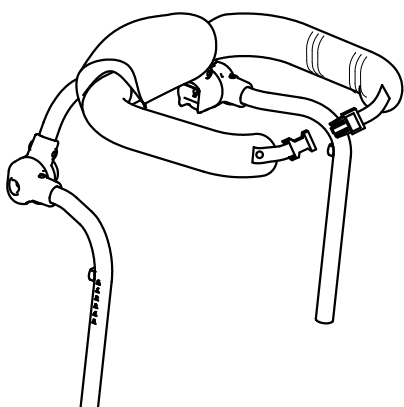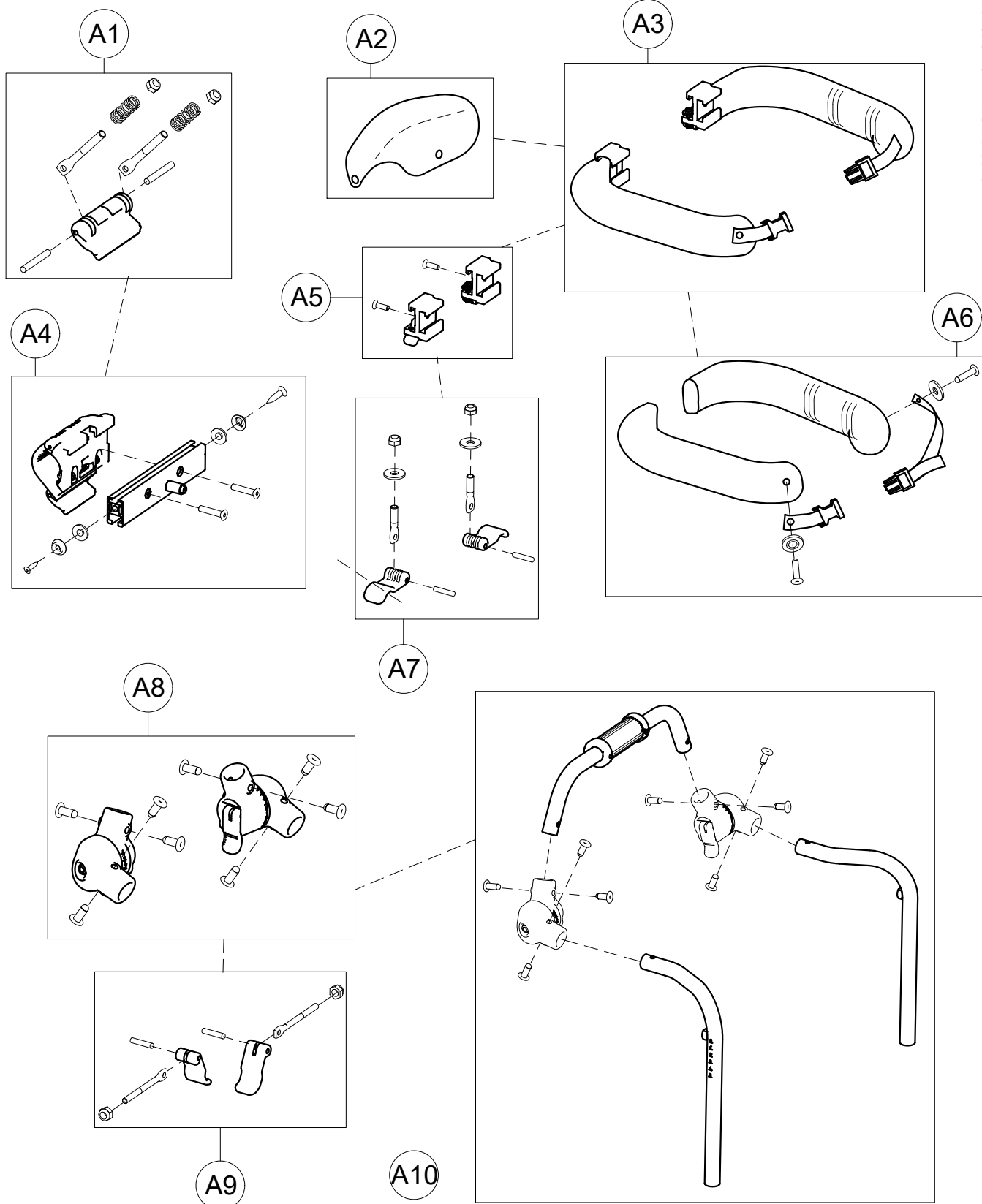

A1	301832	SET OF CLAMPING LEVER TO ROTATE TRUNK SUPPORT
A2	160570P	CENTRAL PADDING
A3	303194	PAIR OF TRUNK HALF SUPPORT
A4	302221	EXTERNAL ROTATION SUPPORT + WIDTH ADJUSTMENT
A5	302298	PAIR OF PROFILE TO SLIDE TRUNK/PELVIC SUPPORT
A6	303221	PAIR OF TRUNK PADDING + STRAPS AND RIVETS
A7	301842	SET OF LEVERS FOR TRUNK SUPPORT WIDTH
A8	303223	PAIR OF TRUNK SUPPORT JOINTS
A9	301843	SET OF TRUNK SUPPORT JOINT LEVERS
A10	303410	FRAME+JOINTS TRUNK SUPPORT

POS. PART NR. DESCRIPTION



### Spare Parts TRUNK SUPPORT size Small

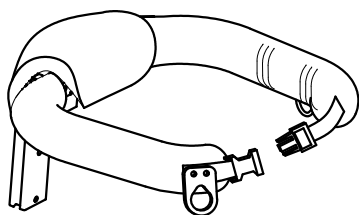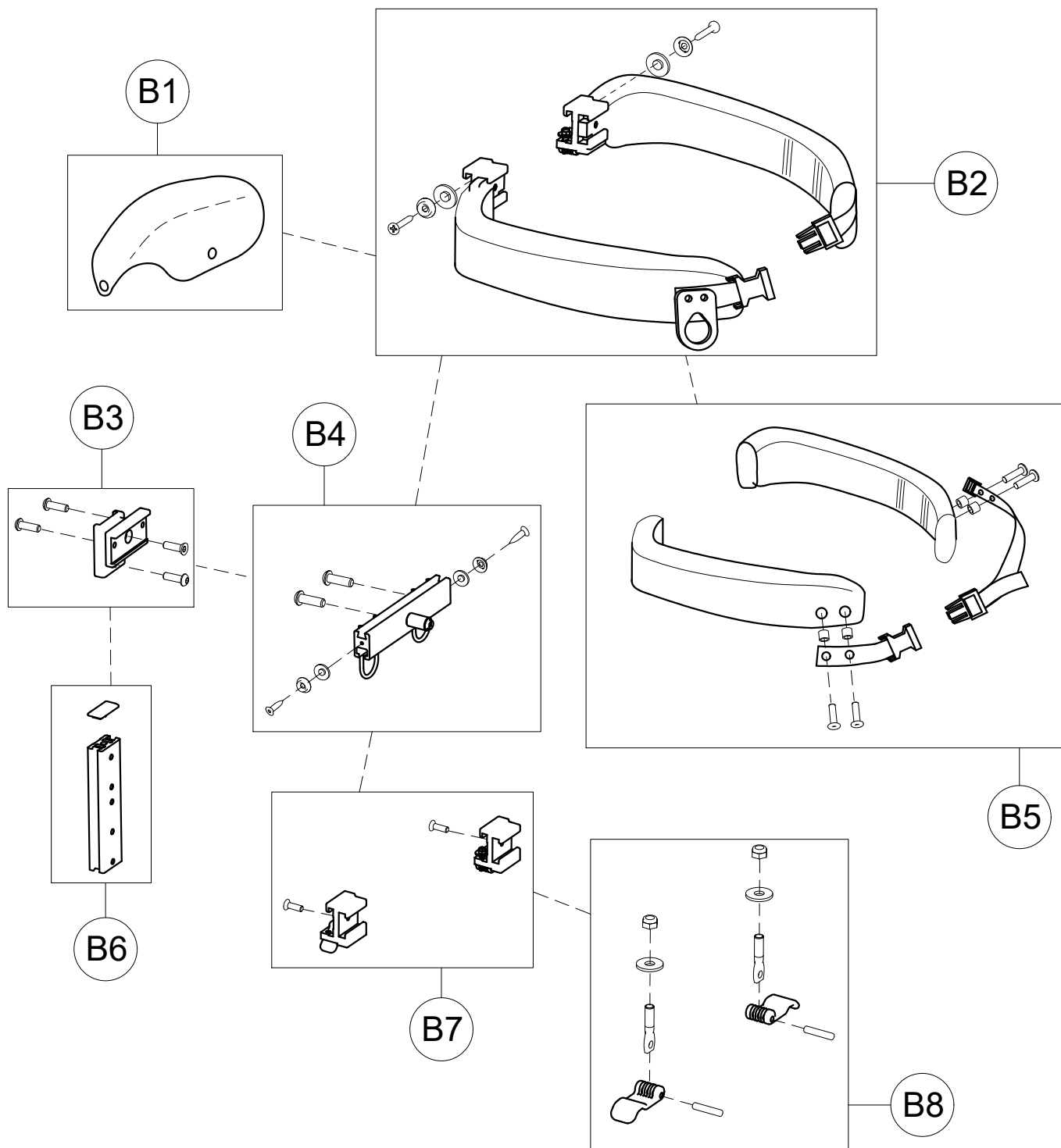

B1	160570P	CENTRAL PADDING
B2	303193	PAIR OF PELVIC HALF SUPPORT
B3	303226	PELVIC SUPPORT +SCREWS
B4	303227	PROFILE TO ADJUST PELVIC SUPPORT IN WIDTH + HOOKS
B5	303229	PAIR OF PELVIC PADDING + STRAPS AND RIVETS
B6	301847	PELVIC SUPPORT HEIGHT ADJUSTMENT
B7	302298	PAIR OF PROFILE TO SLIDE TRUNK/PELVIC SUPPORT
B8	301842	SET OF LEVERS FOR TRUNK SUPPORT WIDTH

POS. PART NR. DESCRIPTION



### Spare Parts PELVIC SUPPORT Small

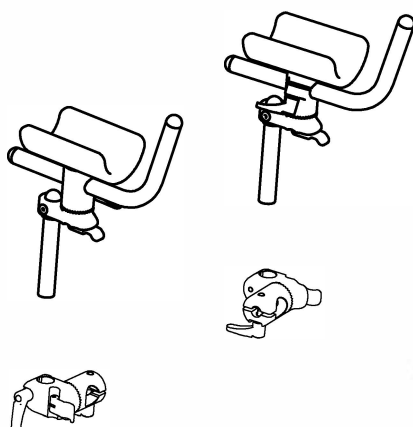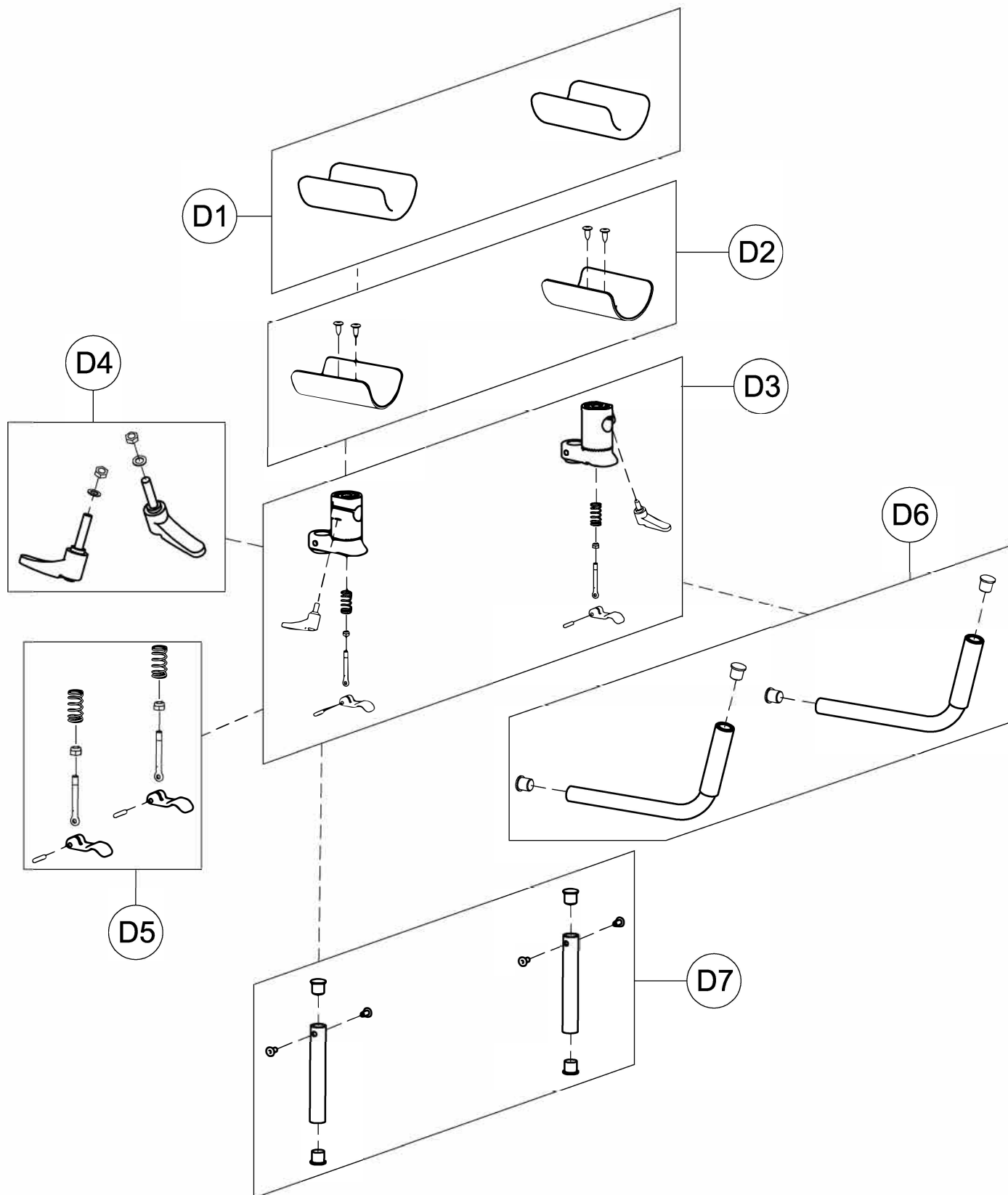

D1	171186	809 ARM SUPPORT UPHOLSTERY
D2	301849	PAIR OF CURVED PLATE 809 small
D3	301851	PAIR OF LOWER/UPPER TOOTHED SUPPORTS 809
D4	303108	PAIR OF M6X25 LEVERS
D5	301852	SET OF TOOTHED SUPP.LEVERS 809
D6	301853	PAIR OF HANDLEBAR BARS 809
D7	301854	PAIR OF VERT.BARS 809 Small



POS.	CODICE	DESCRIZIONE
------	--------	-------------

**Spare Parts ARM SUPPORTS size Small**

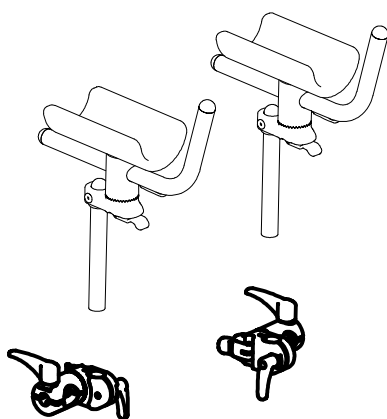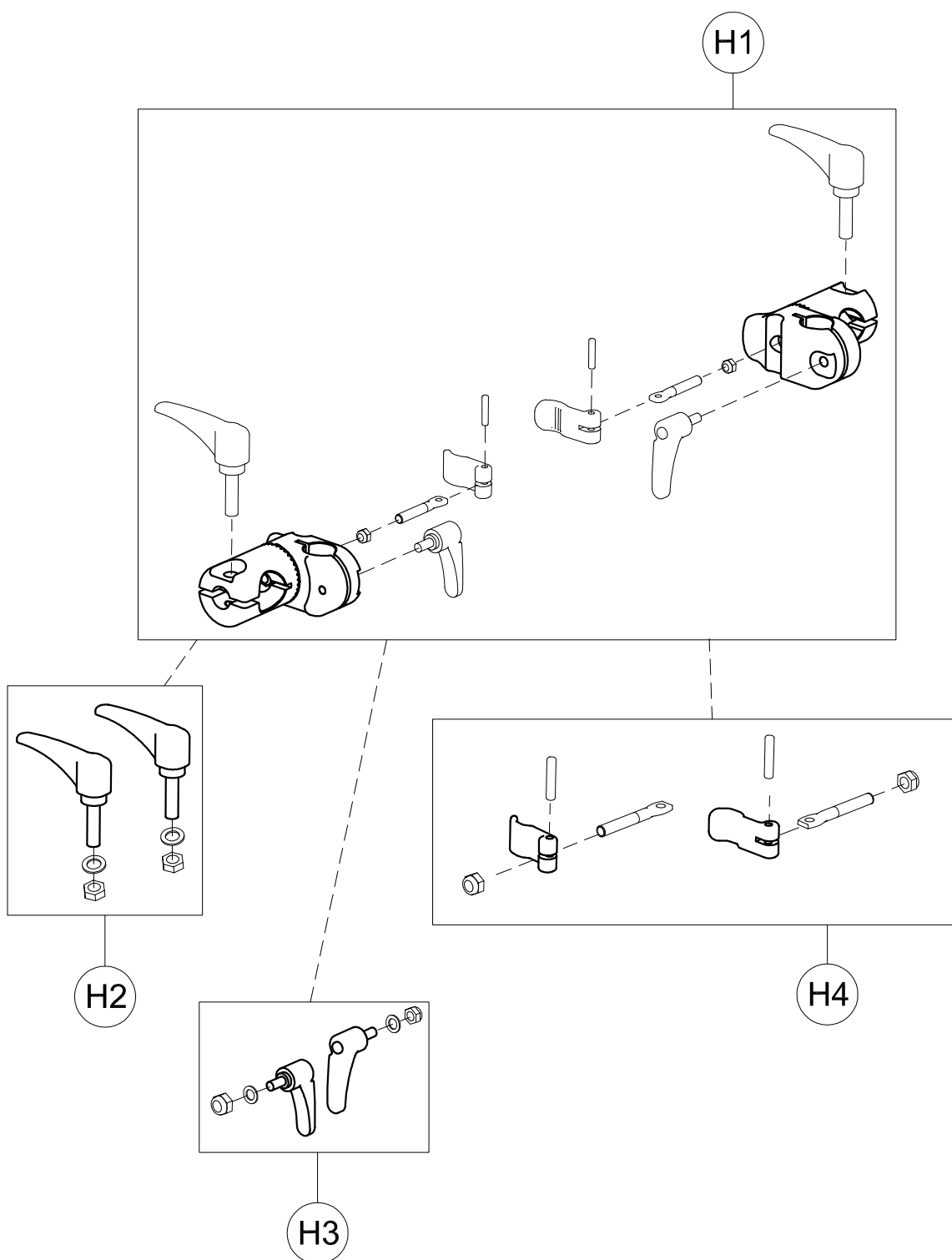

H1	302450	RGT-LEFT SUPPORTS OF 809 GRILLO
H2	301799	PAIR OF M8X35 HANDLES
H3	301800	PAIR OF M6X30 HANDLES
H4	301801	SET OF LEVERS TO LOCK THE HANDLEBAR VERTICALLY
POS.	PART NR.	DESCRIPTION

Spare Parts **ARM SUPPORT JOINT FOR POSTERIOR GRILLO**  
size Small, Medium and Large

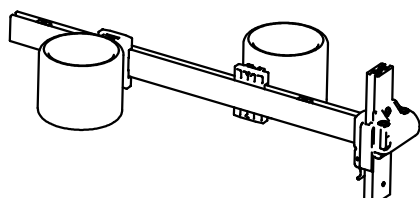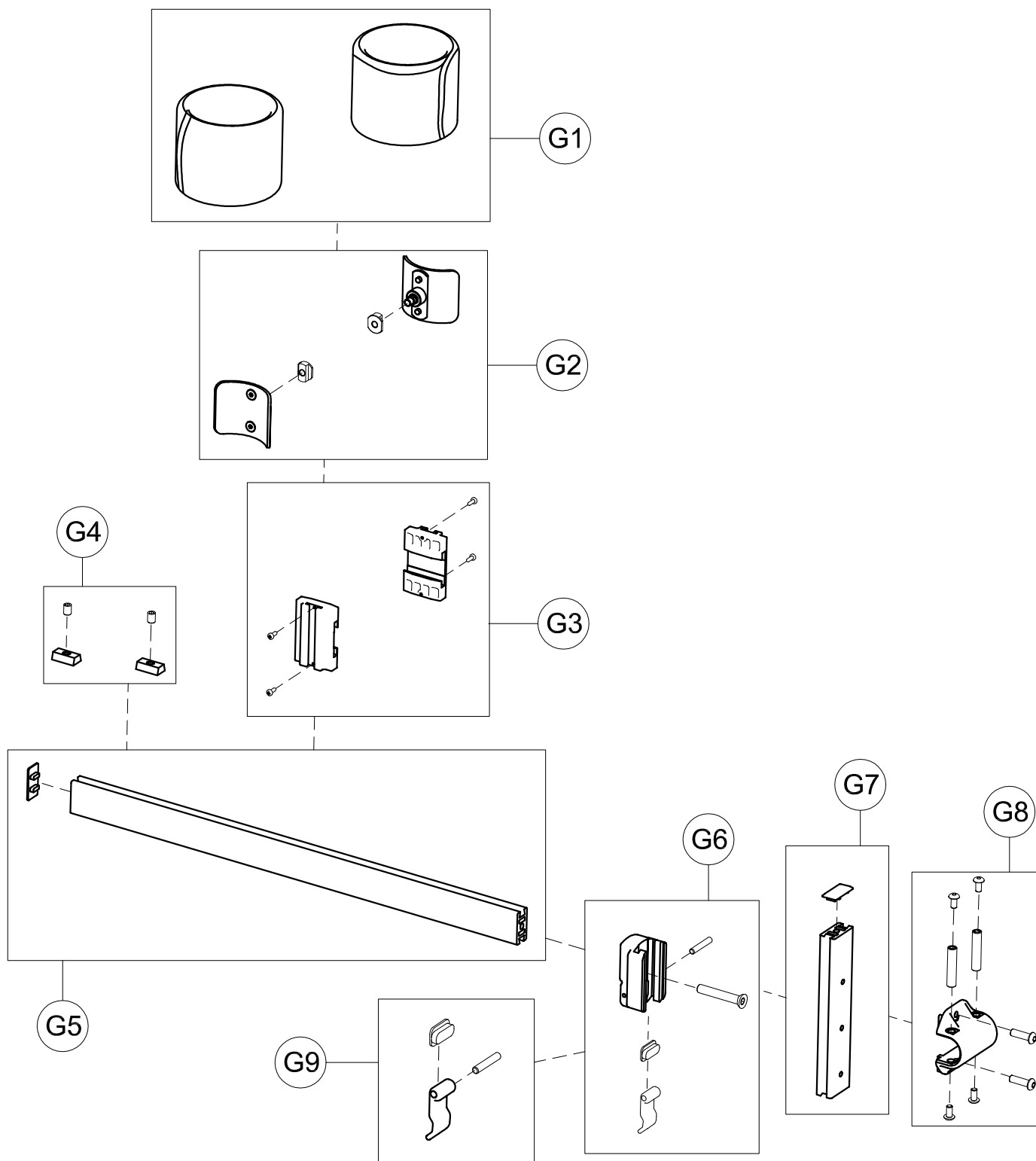

G1	171206	890 small UPHOLSTERY for GRILLO
G2	301856	PAIR OF CURVED PLATE 890 small
G3	301858	PAIR OF ABDUCT.SLIDING PROFILE
G4	301859	PAIR OF ABDUCT. 890 STOP
G5	301860	HORIZONTAL BAR 890 small +CAP
G6	301862	890 FIXING BAR +LEVER/SCREWS
G7	303231	890 VERTICAL .BAR+CAP
G8	303233	FIXING LOCK 890 + SCREWS
G9	301866	SET OF HEIGHT ADJUSTMENT LEVER 890 GRILLO
POS.	PART NR.	DESCRIPTION



Spare Parts LEG DIVIDER+ABDUCTORS size Small

pag. 9

**ORMESA®**



**GRILLO**

THIS DRAWING BELONGS TO ORMESA: IT IS FORBIDDEN TO REPRODUCE, CHANGE OR INFORM ANY THIRD PARTY ABOUT ITS CONTENTS

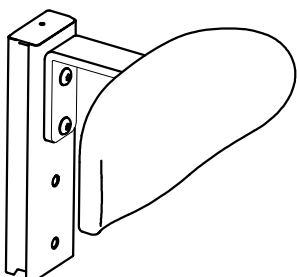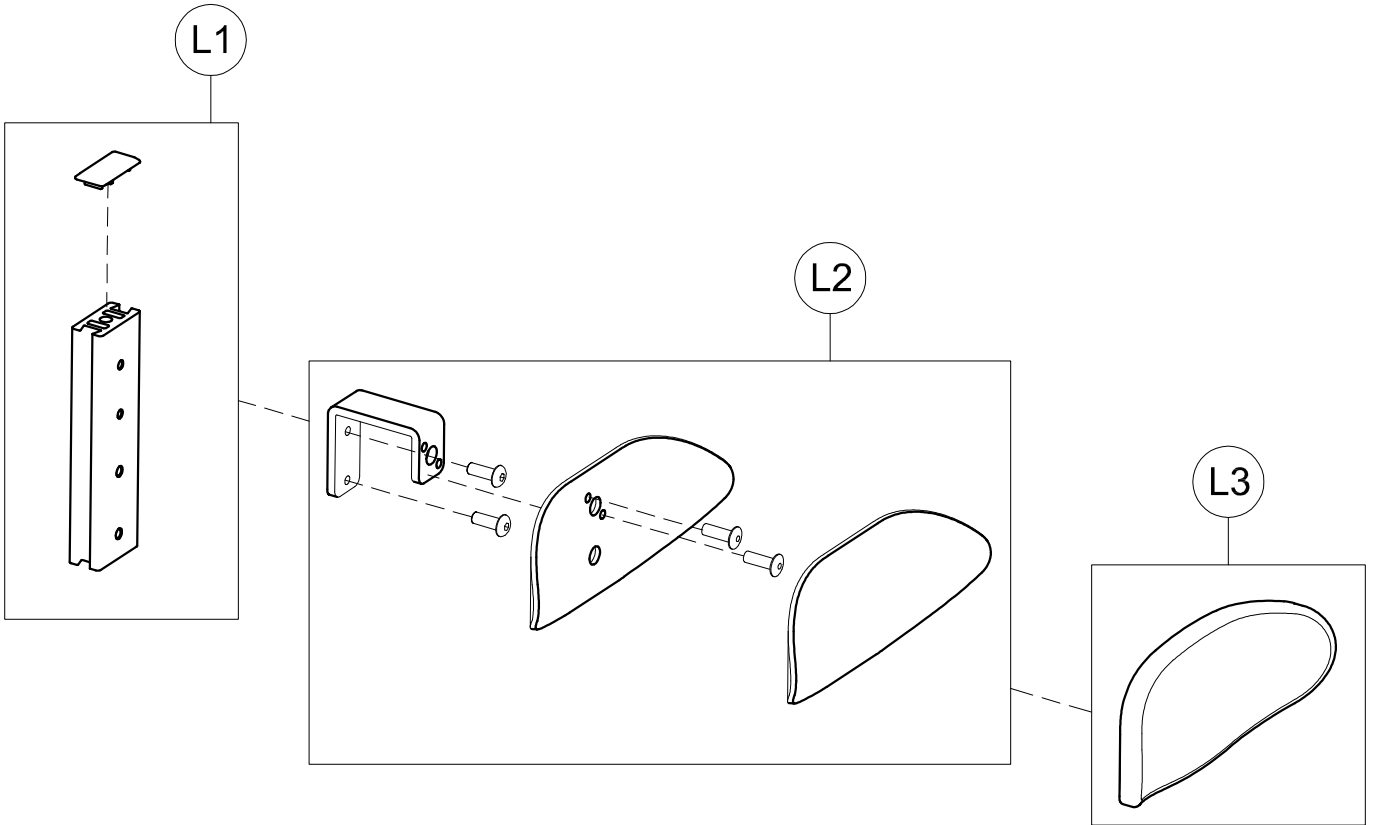

POS.	PART NR.	DESCRIPTION
L1	301867	LUMBAR SUPPORT VERTICAL BAR+CAP
L2	301868	PLATE+LUMBAR SUPPORT CONNECTION
L3	171081	BACK SUPPORT UPHOLSTERY

Spare Parts LUMBAR SUPPORT size Small

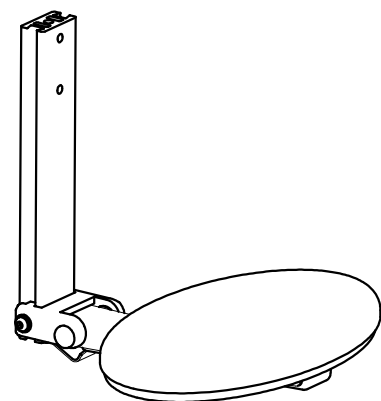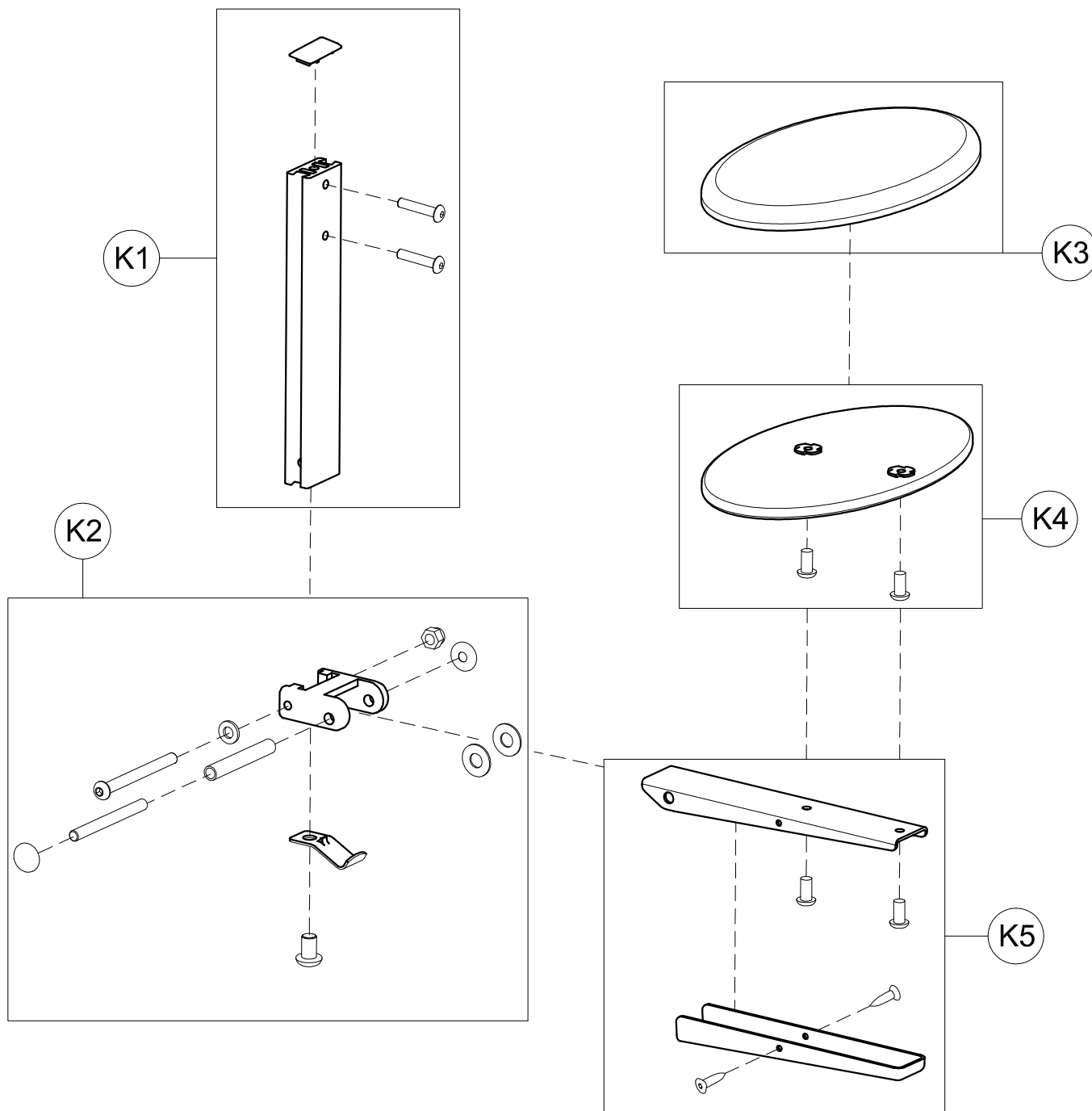

POS.	PART NR.	DESCRIPTION
K1	301871	SEAT VERTICAL BAR+CAP
K2	301873	SEAT FORK+SPRING GRILLO
K3	171278	SEAT COVER FOR 918 Small
K4	301874	WOOD SEAT WITH SCREWS+SPIDERS
K5	301876	SEAT SUPPORT + COVER GRILLO

Spare Parts FOLDING SEAT size Small