

**9.1 - ROLLATORS**

**9.1.1 - STELLA ROLLATOR**

- STELLA ROLLATOR ..... 9.1.1.1 A
- STELLA ROLLATOR ..... 9.1.1.1 B
- STELLA ROLLATOR ..... 9.1.1.2 A
- STELLA ROLLATOR ..... 9.1.1.2 B
- BACKREST ..... 9.1.1.3
- ARMREST ..... 9.1.1.4
- FOOTREST ..... 9.1.1.5
- BRAKES ..... 9.1.1.6
- LOCKING LEVER FOR HEIGHT ADJUSTMENT ..... 9.1.1.7
- CANE HOLDER ASSEMBLY ..... 9.1.1.8
- CUP HOLDER ..... 9.1.1.9

**LEGEND**

**CURRENT** 

• These parts and assemblies are currently being produced and delivered.\*

**SERVICEABLE PARTS** 


• These parts and assemblies are not currently being produced or delivered on new chairs, but they are available as replacement service parts.\*

**DISCONTINUED** 

• These parts and assemblies are no longer being produced or delivered on new chairs, and they are not available as replacement service parts.\*


\* Please contact Technical Support for information on the replacement parts and assemblies.

STELLA ROLLATOR - SECTION 9.1.1

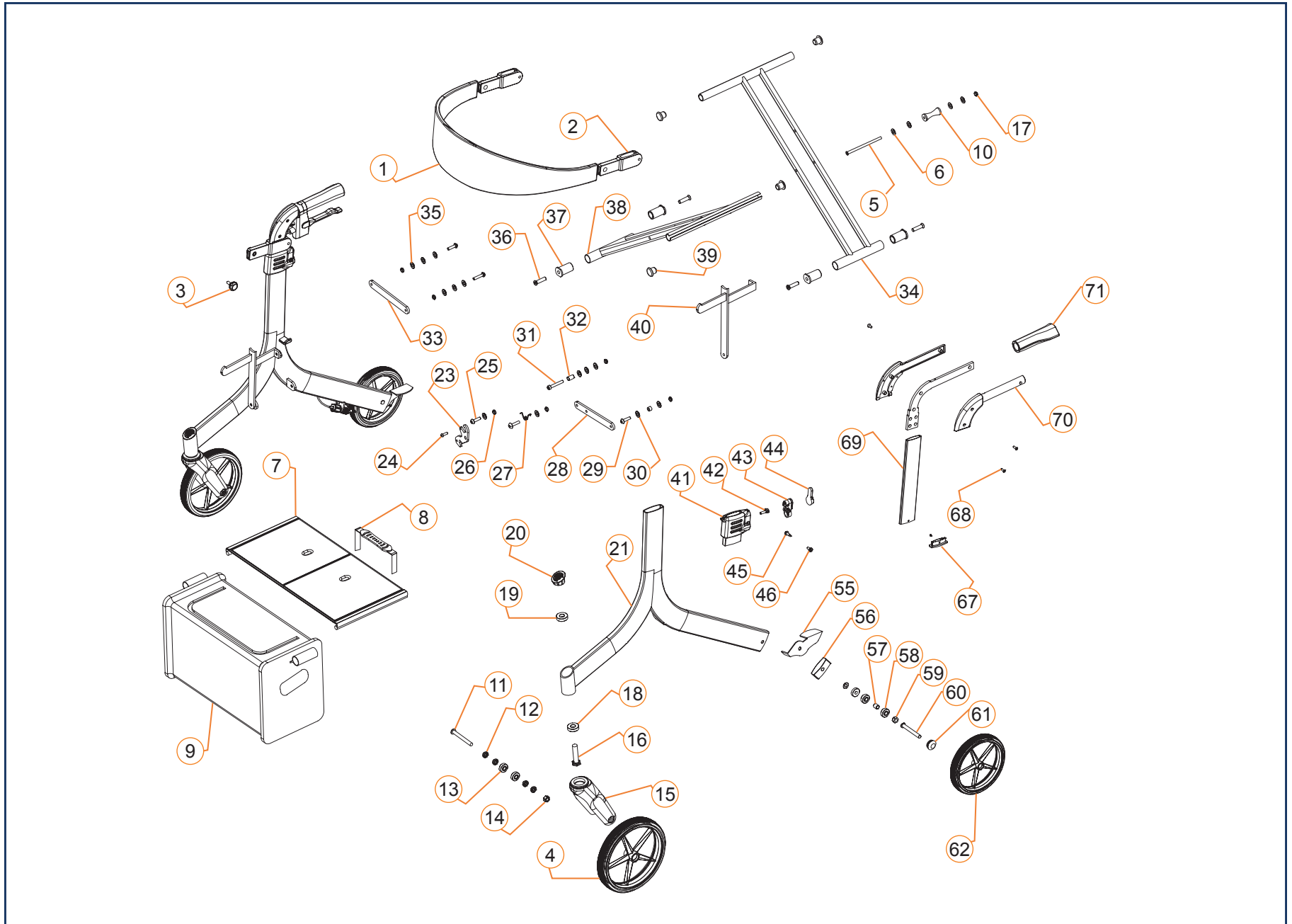

STELLA ROLATOR			
PART #	ITEM #	DESCRIPTION	PRIOR TO
1A,2,3	1001287	BACK STRAP ASSEMBLY STD	2021-09-01
1B,2,3	1000943	BACK STRAP ASSEMBLY HD	2021-09-01
3	1000944	PLASTIC KNOB FOR BACK STRAP -GRAY	2021-09-01
4,13	1001033	8" WHEEL ASSEMBLY - FRONT	2021-09-01
7A,39	1000952	SEAT SLING FOR STELLA - STD 17.5" WIDE, INCL. HDW	2021-09-01
7B,39	1000954	SEAT SLING FOR STELLA - HD 19.5" WIDE, INCL. HDW	2021-09-01
8	1000955	RELEASE SEAT STRAP	2021-09-01
8,23A-27	1000956	RELEASE SEAT STRAP TO UNLOCK STELLA, 18"	2021-09-01
8,23B-27	1000965	RELEASE SEAT STRAP TO UNLOCK STELLA, 20"	2021-09-01
8,23C-27	1000978	RELEASE SEAT STRAP TO UNLOCK STELLA, 23"	2021-09-01
9A	1001291	STELLA BAG STD	2021-09-01
9B	1001292	STELLA BAG HD	2021-09-01
15,16,18-20	1001053	FRONT FORK ASS'Y	2021-09-01
11,12,14	1001102	FRONT WHEEL AXLE ASS'Y	2021-09-01
20	1001209	FRONT CASTER 'CG' CHROME CAP	2021-09-01
39	1001245	CROSS-BRACE END CAP	2021-09-01
41-46	1001288	PUSH HANDLE ADJUSTMENT BUTTON ASSEMBLY	2021-09-01
55A	1001289	CURB CLIMBER, LEFT	2021-09-01
55B	1001290	CURB CLIMBER, RIGHT	2021-09-01
57,58,62	1001208	8" WHEEL ASSEMBLY - REAR	2021-09-01
71	1001255	ERGONOMIC HAND GRIPS INCLUDES REFLECTOR	2019-09-27
NOT SHOWN	1001295	END RUBBER ON BAG SUPPORT	2021-09-01


STELLA ROLLATOR - SECTION 9.1.1

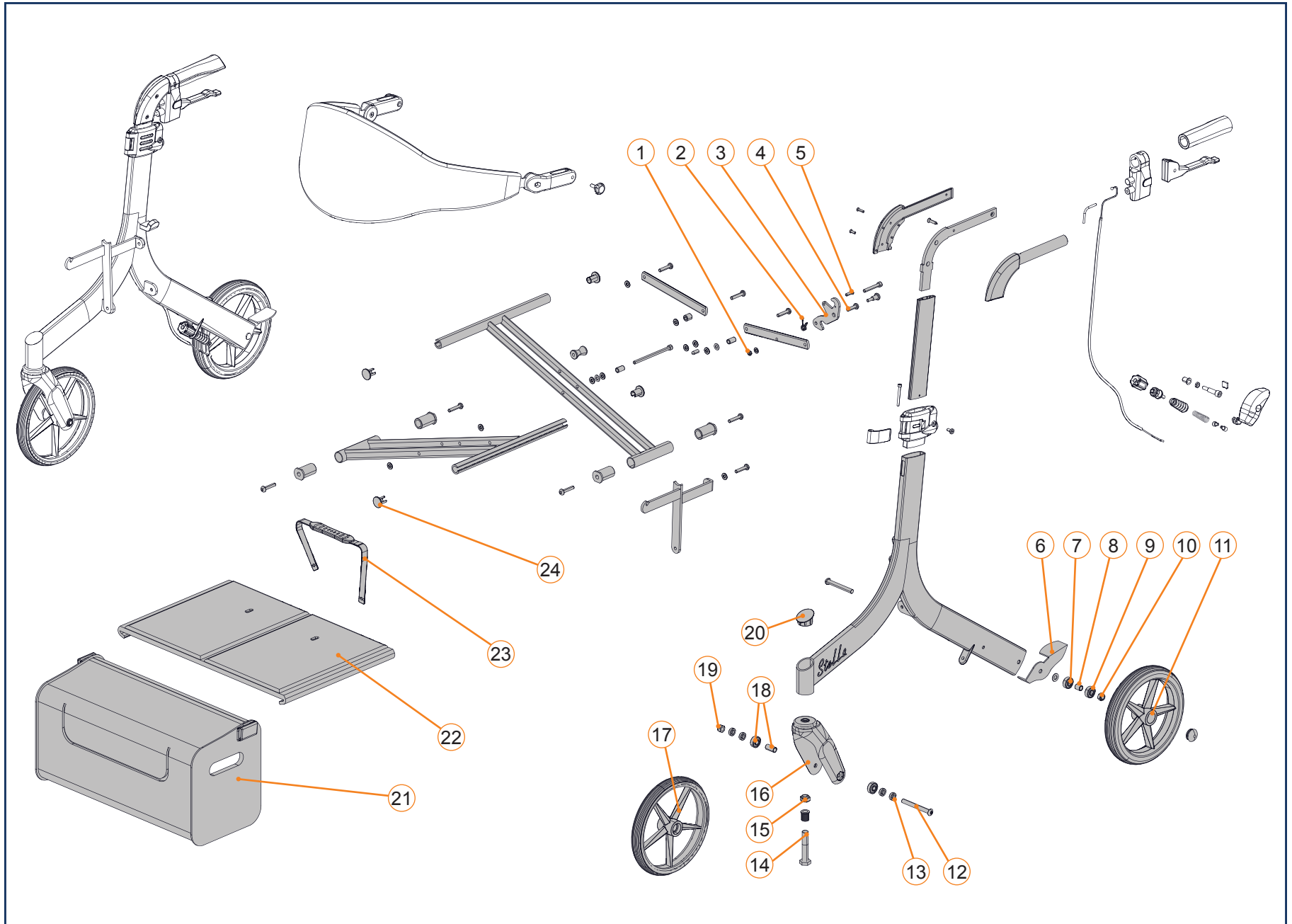

STELLA ROLATOR			
PART #	ITEM #	DESCRIPTION	EFFECTIVE
1-3A,4,5,23	1000956	RELEASE SEAT STRAP TO UNLOCK STELLA, 18"	2021-09-01
1-3B,4,5,23	1000965	RELEASE SEAT STRAP TO UNLOCK STELLA, 20"	2021-09-01
1-3C,4,5,23	1000978	RELEASE SEAT STRAP TO UNLOCK STELLA, 23"	2021-09-01
6A	1001289	CURB CLIMBER LEFT	2021-09-01
6B	1001290	CURB CLIMBER RIGHT	2021-09-01
7-11	1001208	8" WHEEL ASSEMBLY - REAR	2021-09-01
12,13,19	1001102	FRONT WHEEL AXLE ASS'Y	2021-09-01
14-16, 20	1001053	FRONT FORK ASS'Y	2021-09-01
17,18	1001033	8" WHEEL ASSEMBLY - FRONT	2021-09-01
20	1001209	CHROME CAP FOR FRONT CASTER STELLA - CG	2021-09-01
21A	1001291	STELLA BAG STD	2021-09-01
21B	1001292	STELLA BAG HD	2021-09-01
22A,24	1000952	SEAT SLING FOR STELLA - STD 17.5" WIDE, INCL. HDW	2021-09-01
22B,24	1000954	SEAT SLING FOR STELLA - HD 19.5" WIDE, INCL. HDW	2021-09-01
23	1000955	RELEASE SEAT STRAP	2021-09-01

STELLA ROLLATOR - SECTION 9.1.1

BACKREST			
PART #	ITEM #	DESCRIPTION	EFFECTIVE
1	1004927	FEM. HDW FOR FOLDING BACK STRAP SUPP. BKT, STELLA	2024-07-19
1-2a	1003413	KIT, FOLDING BACK STRAP ASS'Y, STELLA, STD	2024-07-19
1-2b	1003414	KIT, FOLDING BACK STRAP ASS'Y, STELLA, HD	2024-07-19

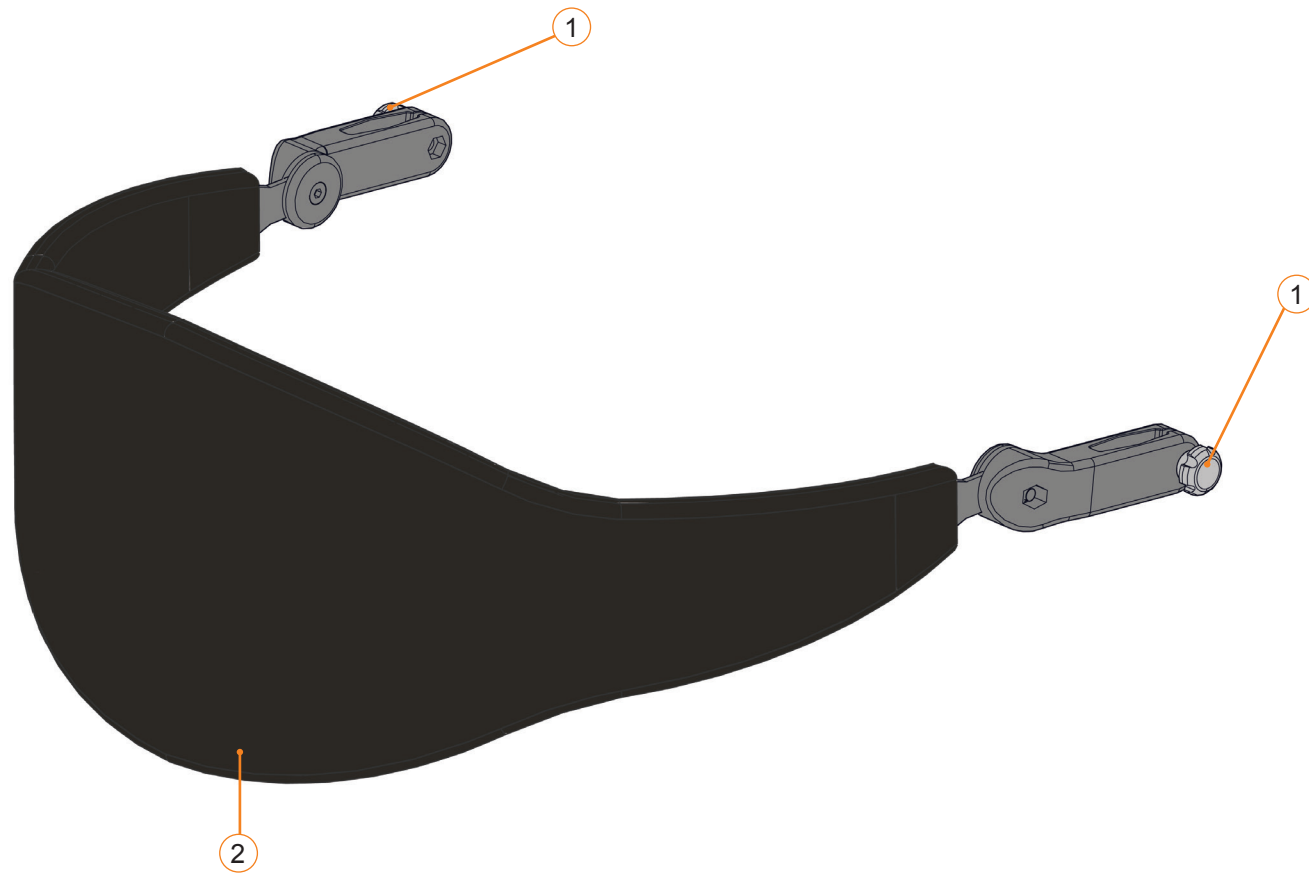


ARMREST			
PART #	ITEM #	DESCRIPTION	EFFECTIVE
1	1003418	TUBULAR ARM SUPPORT PAD	2021-09-01
2	1003419	KIT - ARMREST RELEASE LEVER	2021-09-01
2,3	1003420	COMPLETE ARMREST ASS'Y, LH	2021-09-01
2,4	1003421	COMPLETE ARMREST ASS'Y, RH	2021-09-01







FOOTREST			
PART #	ITEM #	DESCRIPTION	EFFECTIVE
1	1003422	KIT - FOOTREST, LH	2021-09-01
2	1003423	KIT - FOOTREST, RH	2021-09-01
3	1003424	KIT - HEEL LOOPS	2021-09-01
4A	1003425	BLUE FOOTREST ASS'Y (NO FOOTPLATES & ATTACH.) - LH	2021-09-01
4B	1003426	RED FOOTREST ASS'Y (NO FOOTPLATES & ATTACH.) - LH	2021-09-01
4C	1003427	BLK FOOTREST ASS'Y (NO FOOTPLATES & ATTACH.) - LH	2021-09-01
5A	1003428	BLUE FOOTREST ASS'Y (NO FOOTPLATES & ATTACH.) - RH	2021-09-01
5B	1003429	RED FOOTREST ASS'Y (NO FOOTPLATES & ATTACH.) - RH	2021-09-01
5C	1003430	BLK FOOTREST ASS'Y (NO FOOTPLATES & ATTACH.) - RH	2021-09-01

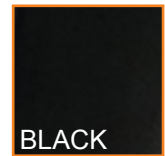



STELLA ROLLATOR - SECTION 9.1.1

BRAKES			
PART #	ITEM #	DESCRIPTION	EFFECTIVE
1	1003433	BRAKE HANDLE, STELLA	2021-09-01
4A	1003939	BRAKE CABLE 18IN, INCLUDES CRIMP	2021-09-01
4B	1003940	BRAKE CABLE 20IN, INCLUDES CRIMP	2021-09-01
4C	1001246	BRAKE CABLE STELLA 23IN, INCLUDES CRIMP	2021-09-01
1A-4A	1003935	KIT, BRAKE HANDLE CABLE ASS'Y, STELLA 18 IN, LH	2021-09-01
1B-4B	1003936	KIT, BRAKE HANDLE CABLE ASS'Y, STELLA 18 IN, RH	2021-09-01
1C-4C	1003937	KIT, BRAKE HANDLE CABLE ASS'Y, STELLA 20 IN, LH	2021-09-01
1D-4D	1003938	KIT, BRAKE HANDLE CABLE ASS'Y, STELLA 20 IN, RH	2021-09-01
1E-4E	1003431	KIT, BRAKE HANDLE CABLE ASS'Y, STELLA 23 IN, LH	2021-09-01
1F-4F	1003432	KIT, BRAKE HANDLE CABLE ASS'Y, STELLA 23 IN, RH	2021-09-01
5A-12A	1001253	BRAKE ASSEMBLY, LEFT	2021-09-01
5B-12B	1001254	BRAKE ASSEMBLY, RIGHT	2021-09-01
13A	1003434	SLOW DOWN BRAKES FOR STELLA ROLLATOR, LH	2021-09-01
13B	1003435	SLOW DOWN BRAKES FOR STELLA ROLLATOR, RH	2021-09-01

LOCKING LEVER FOR HEIGHT ADJUSTMENT			
PART #	ITEM #	DESCRIPTION	EFFECTIVE
1	1003436	LOCKING LEVER FOR PUSH HANDLE HEIGHT ADJ., STELLA	2021-09-01



CANE HOLDER ASSEMBLY			
PART #	ITEM #	DESCRIPTION	EFFECTIVE
1,2	1001293	CANE HOLDER ASSEMBLY	2021-09-01



CUP HOLDER			
PART #	ITEM #	DESCRIPTION	EFFECTIVE
1	1001294	CUP HOLDER	2021-09-01

