


INSTRUCTION GUIDE

GROUP 22/21 BATTERY CONNECTIONS WITH COVERS

GROUPE 22/21 CONNECTIONS

Diagram (A)


The information label for connecting battery cables is located on the inside of the battery access door as shown in image A below.

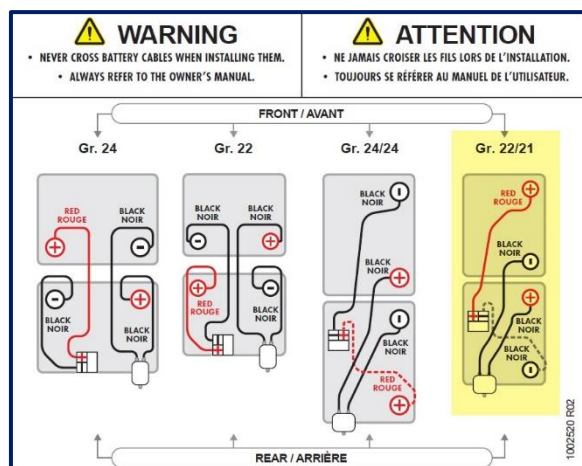

Connection sequence of cables

STEP 1

Slide covers to have access to the terminals and to pull the wires for connection, as shown in image B.

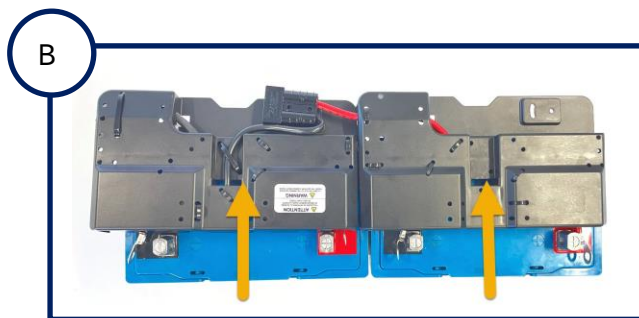

GR. 22 BATTERY CONNECTION (cont'd)
— NOTES —

When connecting the cables to the battery, it is important to follow the sequence of the hardware as shown in the illustration below.



Make sure that the connectors on the terminals remain down so as not to interfere when closing covers, as shown in image below.


STEP 2

Connect breaker cables (1 & 2) and connector cables (3 & 4) according to the number sequence shown in image C.


STEP 3

Lift covers slightly to close over terminals as shown in image D.


STEP 4

The strap must be routed parallel along both batteries over the covers, as shown in image E.

