

INFORMATION BULLETIN

CG Air REPAIR KIT REVISED

INTRODUCTION

The CG Air cushion repair kit has been revised. Effective October 15, 2023, we're replacing the old glue and patches with glue only (no patches required). **Lepage Extreme Glue Gel** will now be used to repair CG Air cushion air leaks.

DETAILS ON THE CHANGE

Lepage Extreme Glue Gel doesn't require patches and is very easy to use. We will keep an in-house inventory of both kits (consumer and institutional/dealer) as shown below. However, you can inform customers that they can purchase the same type of glue in hardware stores under the following brands depending on the location of purchase:



* Note that for the US market, the correct glue is available in one size only (.60 fl oz), a larger (50 g) package can be found in some hardware stores, but the product is not the same.

CONSUMER KIT CONTENT

- Three (3) alcohol Prep Pads
- One tube of Lepage Extreme Glue Gel (20 g)¹
- Marketing code for Order Forms: CGAPK3
- Kit part code: 1001402
- Tube only part code: 1004457



INSTITUTIONAL/DEALER KIT CONTENT

- Fifteen (15) alcohol Prep Pads
- One tube of Lepage Extreme Glue Gel (50 g)¹
- Marketing code for Order Forms: CGAPK15
- Kit part code: 100140
- Tube only part code: 1004457



¹ Lepage Glue: [Safety Data Sheet](#)

CG Air Repair Kit Revised**OTHER IMPLICATIONS****Known limitations**

The glue takes 12 to 24 hours to cure.

Availability and Pricing

Two kits are available, and both are in stock using the codes specified in the previous section. Prices in hardware stores vary according to store and market (Between \$6-\$8 USD or \$10-\$15 CAD/AUD)

Corresponding updates

All Order Forms for CG Air cushions have been updated with the new repair kits and are published on our website. These items will be in the Parts Manual in its next release.

An Instruction Guide (Part code: 1001496) will be included with all repair kit orders. It will also be on our website in the Support/For Pros/Instruction Guides section.