

INFORMATION BULLETIN

NEW SIGNATURE SEAT PAN


INTRODUCTION

Introduced as a running change, we are happy to announce the release of the next components of our newly re-designed Amyseat Signature Series power seating system. The NEW Signature seat pan provides easier slotted adjustment of width and depth while retaining our well-known structural integrity and durability.


The new design reduces the number of adjustment bolts that must be fully removed while also providing an etched guide to show settings for current and adjusted width settings.

SIDE BY SIDE

FORMER SEAT PAN

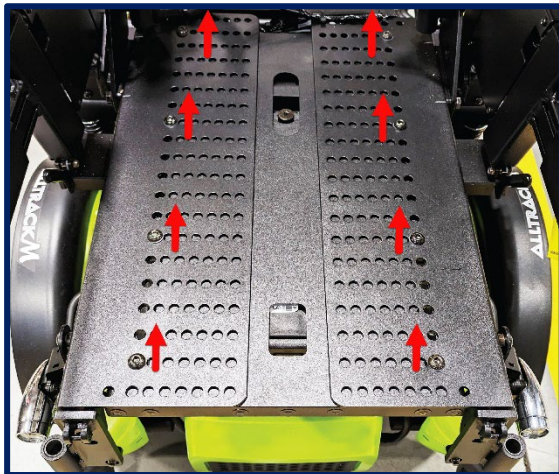

NEW SIGNATURE SEAT PAN

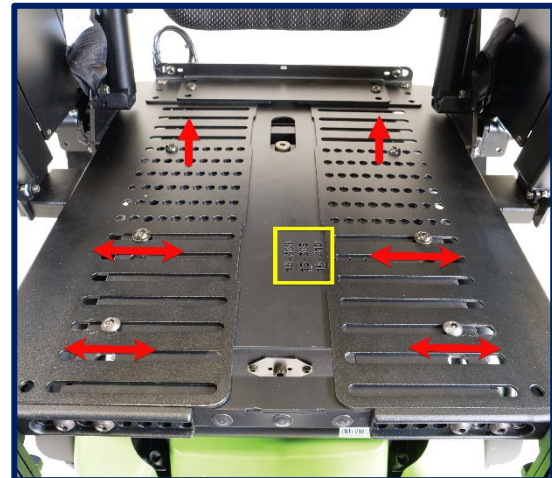

Adjustment

- Adjustment is done by removing all bolts from each of the 2 plates (*suggested to do one side at a time*) and while holding them, move them horizontally to desired width.
- Adjustment can be more time consuming.

Adjustment

- Adjustment is done by loosening all bolts in the slots without removing them, then unscrewing and completely removing the ones in the holes.
- This method keeps both plates in position during adjustment.
- A measurement guide in inches and millimeters is engraved on the base of the pan.

New SIGNATURE Seat Pan (cont'd)**OTHER CONSIDERATIONS****Availability and Pricing**

The new Signature seat pan is currently being installed on all Alltrack models as the old stock for each size runs out. The cost of this seat pan will not change the price of the wheelchair on the Order Form.

Corresponding updates

The Order Forms have not changed and remain current on our website. The new Signature seat pan is in the Parts Manual in section 2.2.1.

An Instruction Guide for adjustments is on our website in the Support/For Pros/Instruction Guides section under the following name:

ADJUSTING THE NEW SIGNATURE SEAT PAN, BACK, CENTER OF GRAVITY (2024)