


A1	106831	SEAT UNIT BUG Small LIGHT GREY GREEN UPHOLSTERY
A2	106832	SEAT UNIT BUG Small DARK GREY GREEN UPHOLSTERY
A3	106833	SEAT UNIT BUG Small LIGHT GREY PINK UPHOLSTERY
A4	106834	SEAT UNIT BUG Small DARK GREY PINK UPHOLSTERY
B	106829	869 Small FOUR WHEEL BASE FOR BUG BLACK FRAME
D	108117	856 Small HIGH LOW ADJUST BASE BUG
POS.	PART NR.	DESCRIPTION

### Spare Parts BUG size Small

A5

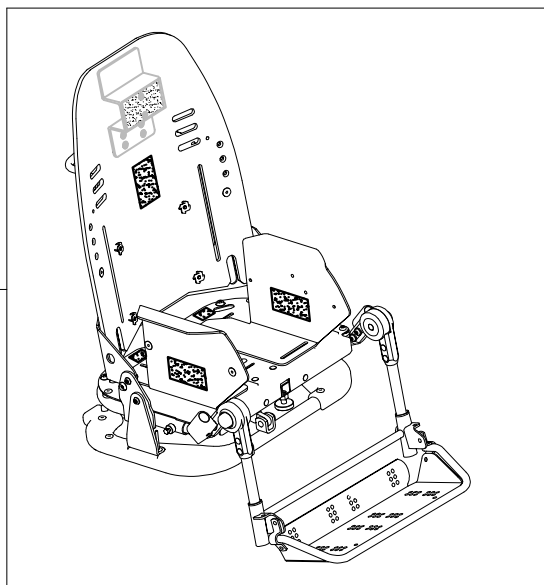

A41




A42



A43





A44

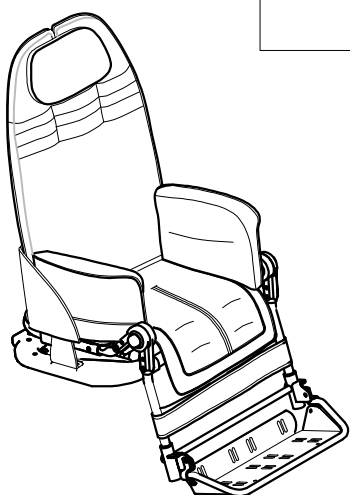

A5	105601	BUG Small SEAT UNIT (NO UPHOLSTERY)
A41	171641	LIGHT GREY GREEN UPHOLSTERY BUG Small
A42	171642	LIGHT GREY PINK UPHOLSTERY BUG Small
A43	171643	DARK GREY GREEN UPHOLSTERY BUG Small
A44	171644	DARK GREY PINK UPHOLSTERY BUG Small
POS.	PART NR.	DESCRIPTION

**Spare Parts BUG Small SEAT UNIT**









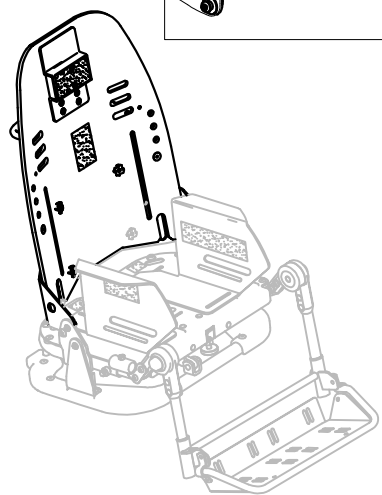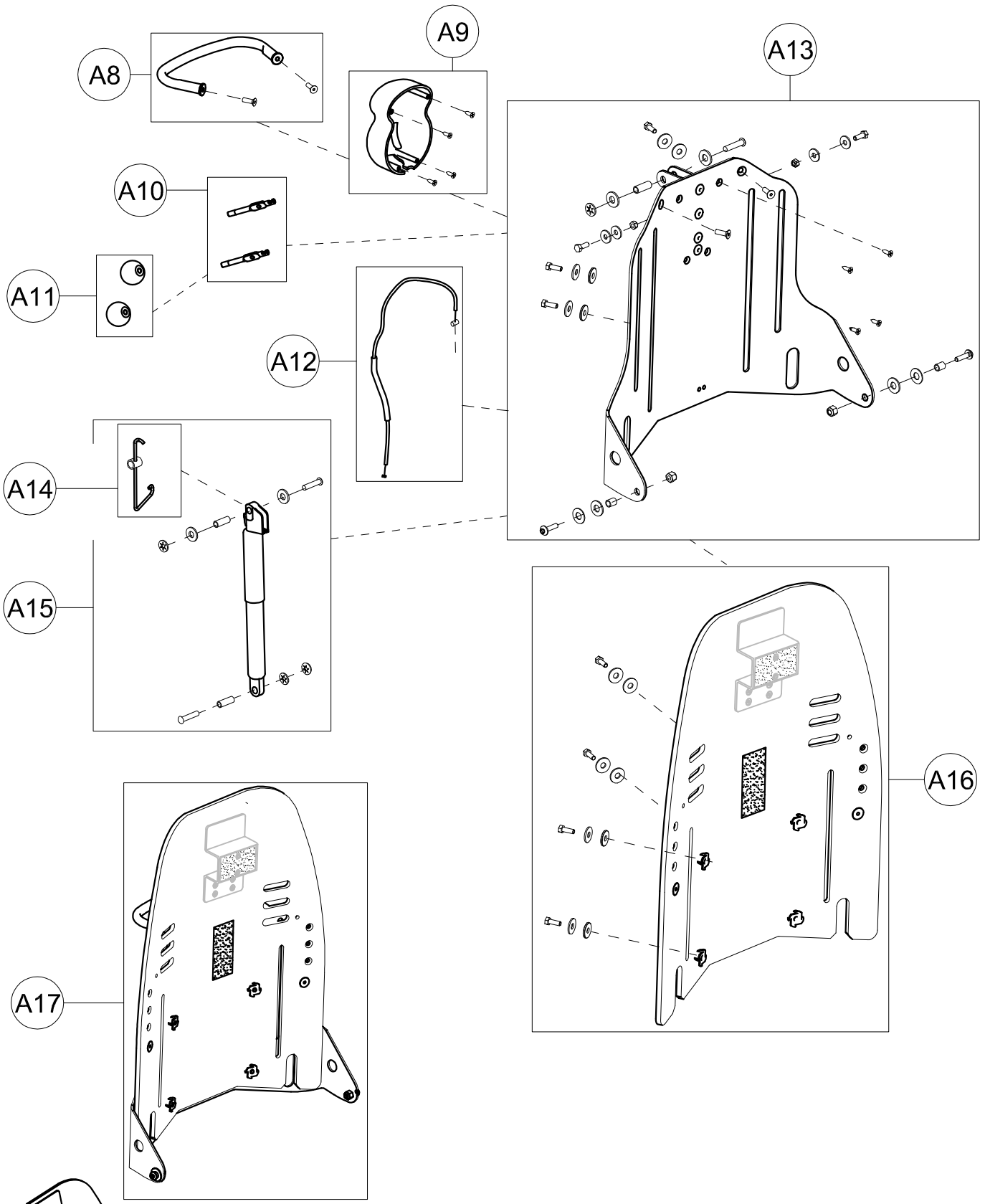

A8	301355	COMPLETE HANDLE OF THE SEATING UNIT FOR BUG
A9	301356	COMPLETE COVER OF THE BACKREST FOR BUG
A10	301965	PAIR OF INCLINATION/TILTING CONTROL LEVERS BUG
A11	301966	PAIR OF KNOB D30 RED / BLACK BUG
A12	21350	CABLE + COMPLETE SHEATH OF THE SEAT FOR BUG Small
A13	301361	COMPLETE ADJUSTMENT OF THE BACKREST FOR BUG Small
A14	301969	SPRING, Ø0, 7 + BUSHING FOR GAS SPRING
A15	301967	COMPLETE GAS SPRINGS BACKREST FOR BUG Small
A16	303348	BACKREST + SCREWS FOR BUG Small
A17	303350	COMPLETE BACKREST BUG Small
POS.	PART NR.	DESCRIPTION


Spare Parts BACKREST STRUCTURE mis. Small



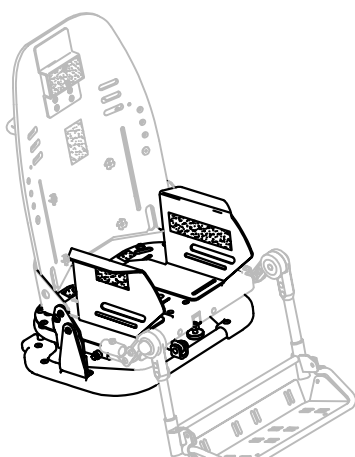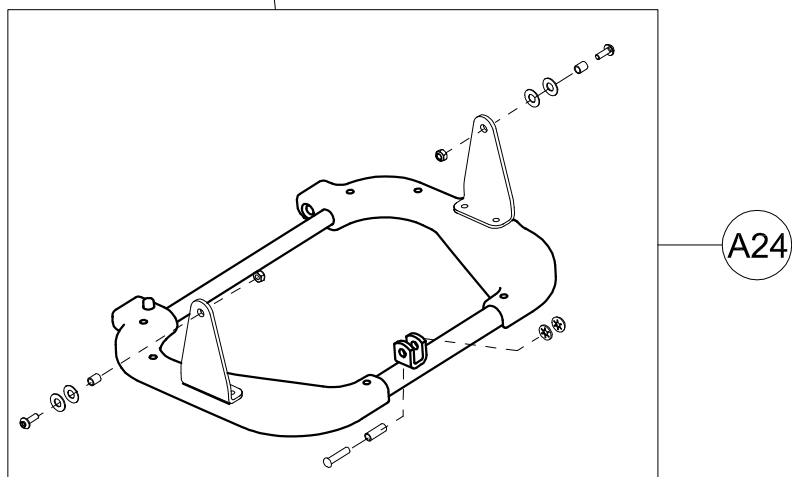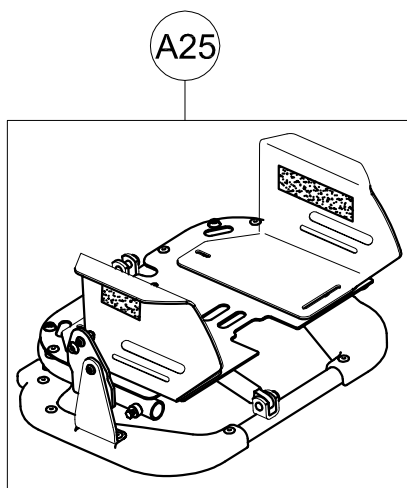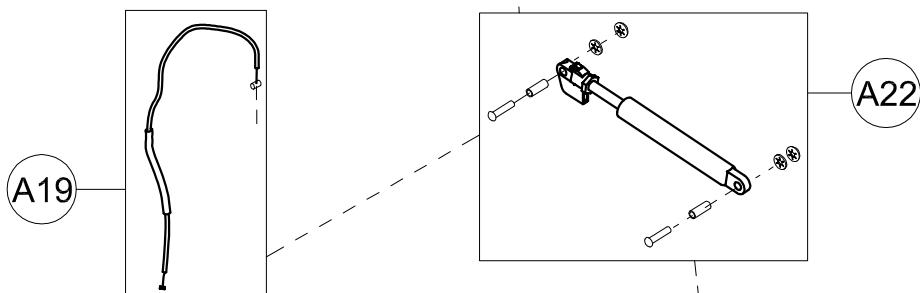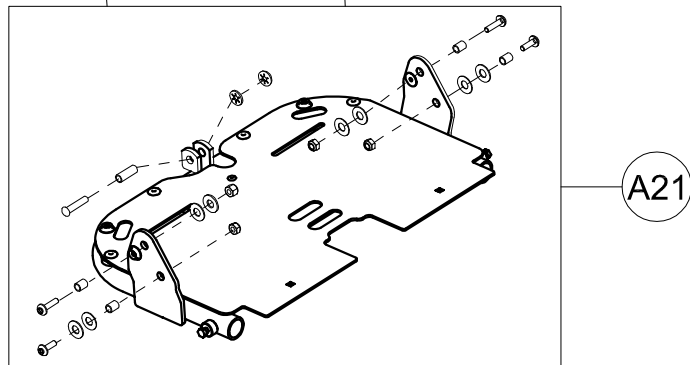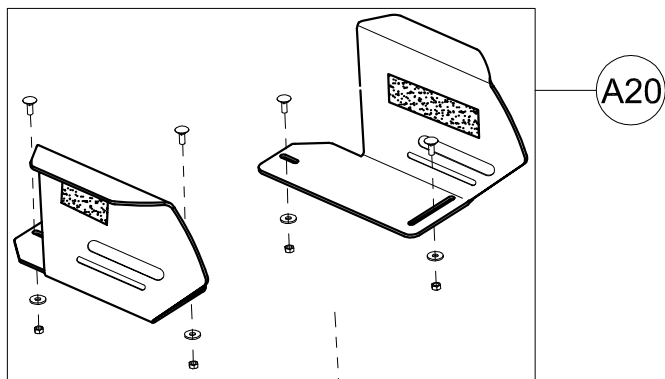

A19	21350	CABLE+ SHEATH SEAT BUG Small
A20	302043	COMPLETE RIGHT/LEFT SIDES OF THE SEAT FOR BUG Small
A21	301348	COMPLETE SEAT FOR BUG Small
A22	301587	GAS SPRING FOR BUG TILT + SCREWS
A24	301351	LOWER SEAT FOR BUG Small
A25	301447	COMPLETE LOWER SEATING FRAME BUG Small
POS.	PART NR.	DESCRIPTION

Spare Parts LOWER SEATING FRAME Size Small

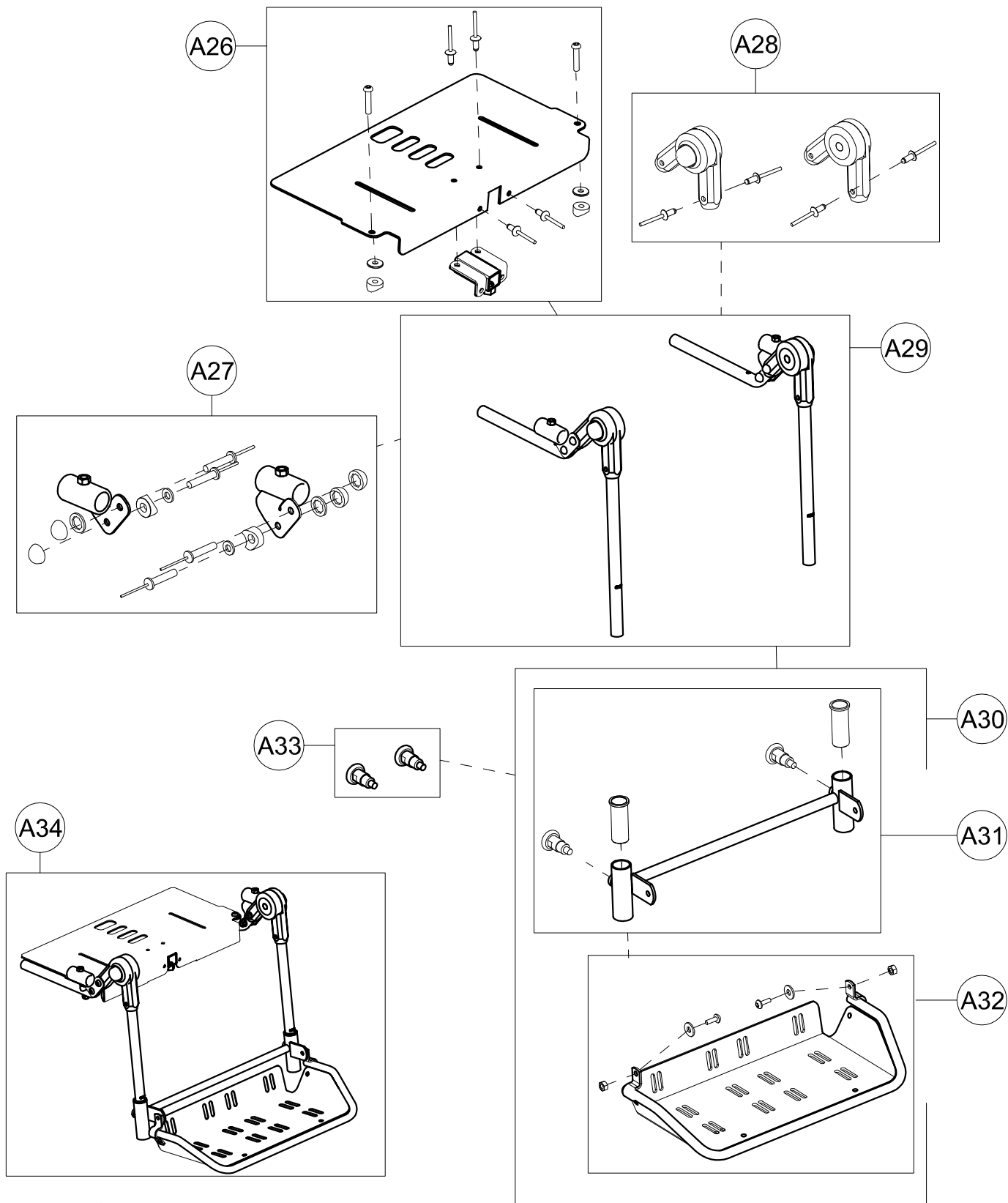

A26	301970	EXTENDED SEAT BUG Small +JOIN PIECE OF THE ABDUCTION BLOCK
A27	303062	PAIR OF TRAY ATTACHMENT BUG
A28	301972	PAIR OF LEGREST PVC JOINTS FOR BUG Small
A29	303262	RIGHT/ LEFT LEGRESTS + JOINTS BUG Small
A30	301435	COMPLETE FOOTREST FOR BUG Small 3R-4R
A31	301437	COMPLETE TROUSER HEM-BINDING FOR BUG Small 3R/4R
A32	301439	COMPLETE FOOTREST BUG Small WITHOUT HEM-BINDING
A33	301892	PAIR OF M10X1 SPRING LOCKING MECHANISM
A34	303264	FOOT AND LEGREST WITH EXTENDED SEAT BUG Small
POS.	PART NR.	DESCRIPTION

Spare Parts


FOOT AND LEGREST WITH  
EXTENDED SEAT size Small


pag. 5

**ORMEDA**

**BUG**

THIS DRAWING BELONGS TO ORMEDA: IT IS FORBIDDEN TO REPRODUCE, CHANGE OR INFORM ANY THIRD PARTY ABOUT ITS CONTENTS

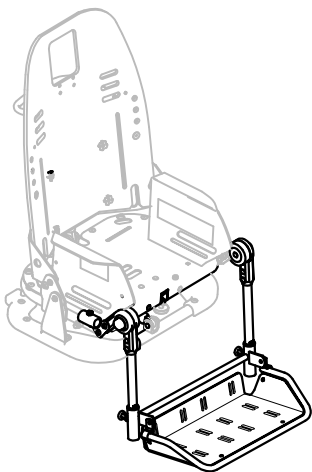

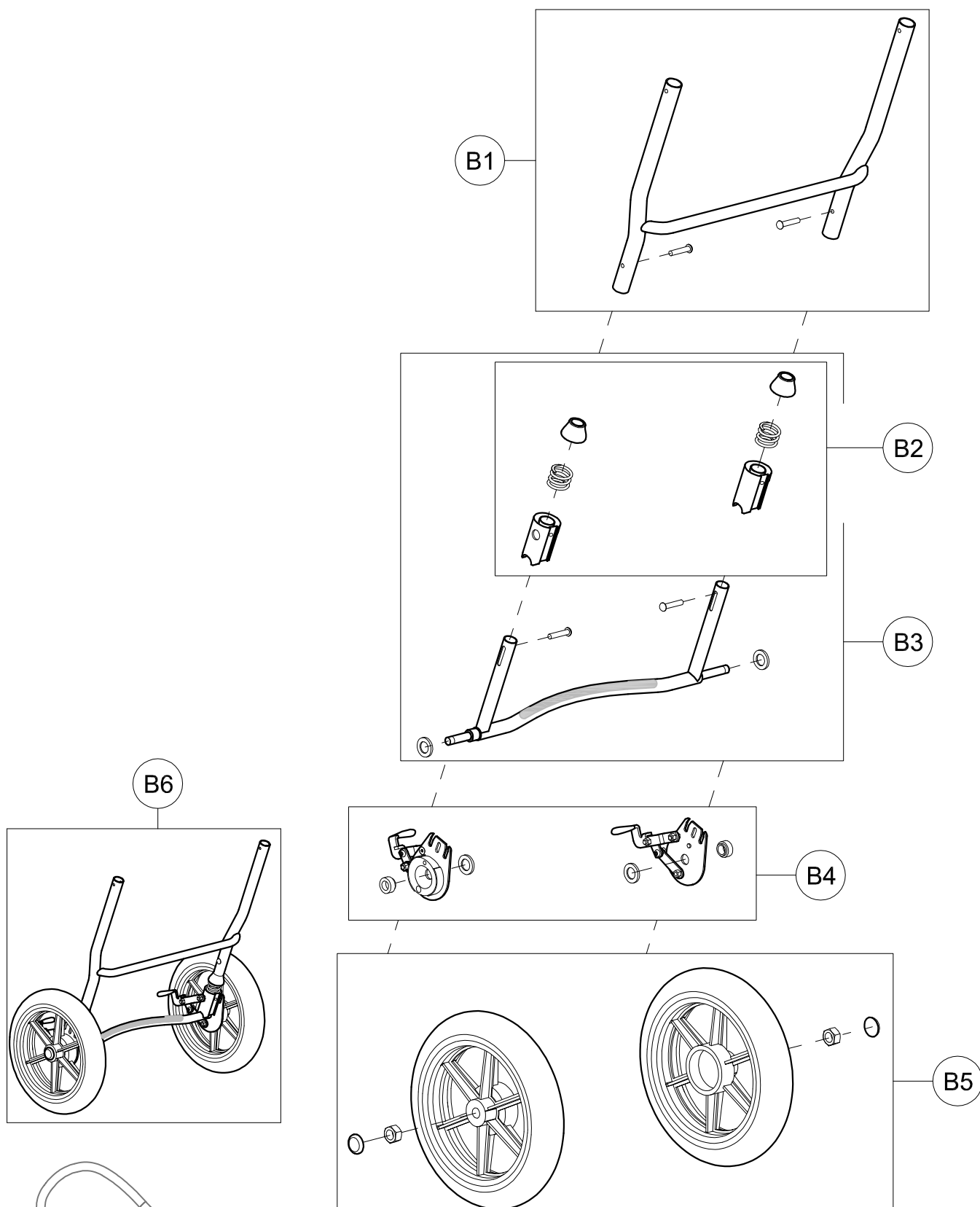


B1	301975N	BACK FRAME FOR BUG Small 4R (BLACK)
B2	301979	SET OF SHOCK ABSORBERS FOR BUG Small
B3	301977N	COMPLETE FRAME OF SHOCK ABSORBER FOR BUG Small 4R
B4	301981	PAIR OF RIGHT-LEFT BRAKE COMPLETE BUG
B5	301982	PAIR OF REAR WHEELS 10" BUG Small
B6	301984N	COMPLETE BACK FRAME OF THE 869 BASE BUG Small
POS.	PART NR.	DESCRIPTION


Spare Parts BACK FRAME OF THE 869 BASE size Small

**ORMESA**


**BUG**

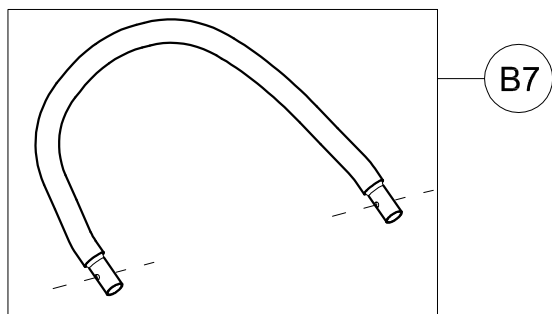


B7

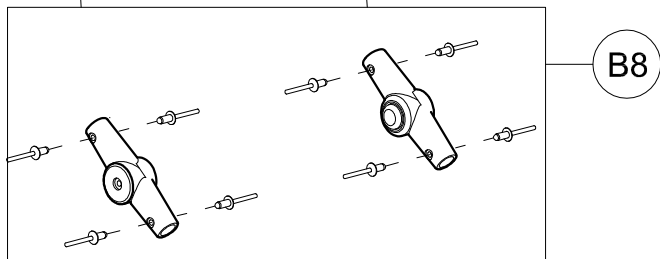


B8

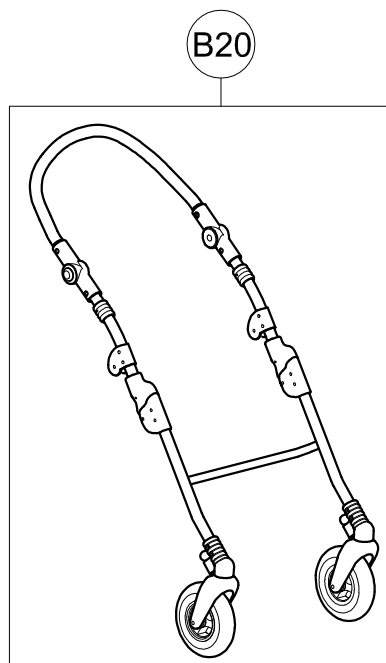


B20

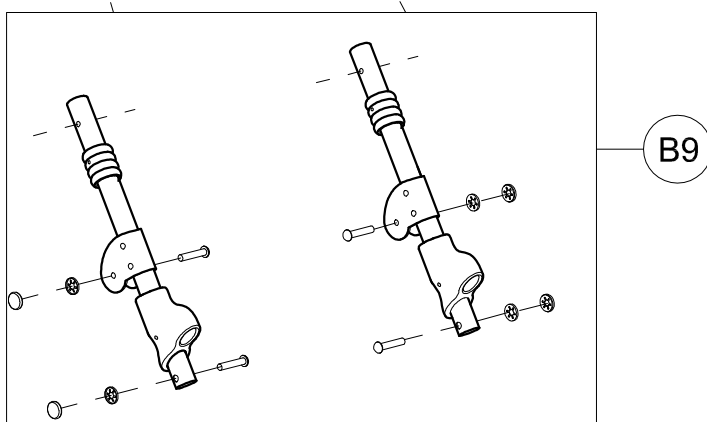


B9

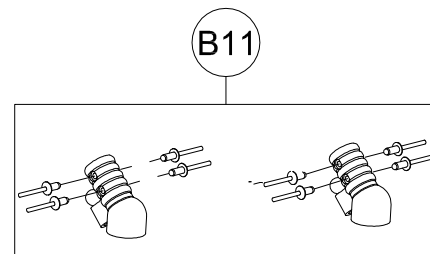


B11

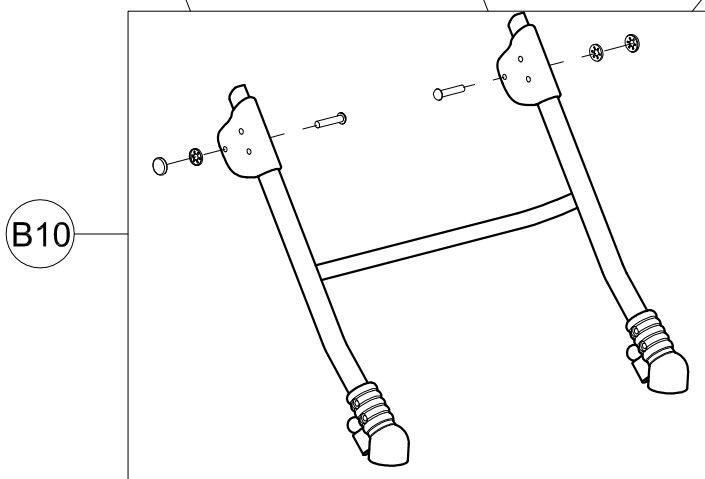


B10

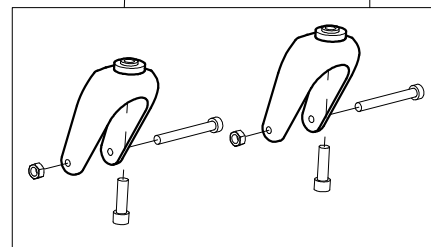

B12

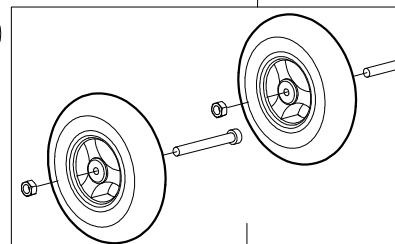






B13

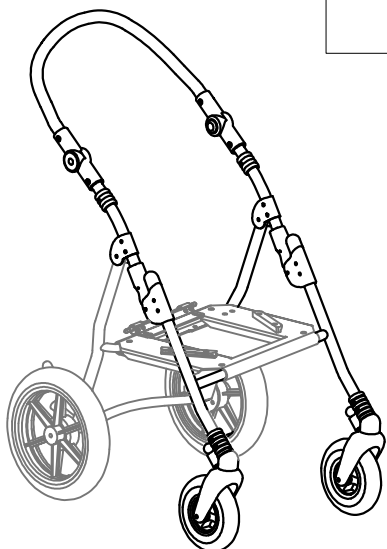

B7	301465	COMPLETE PUSH HANDLE OF BUG Small
B8	302431	PAIR OF PUSH HANDLE JOINTS WITH RIVET BUG 3R/4R
B9	301987N	PAIR OF PUSH HANDLE TUBES FOR BUG Small 4R
B10	301989N	COMPLETE FRONT FRAME + WHEEL SUPPORT BUG Small 4R (BLACK)
B11	301991	PAIR OF FRONT WHEEL SUPPORTS FOR BUG 4R
B12	302262	PAIR OF FRONT WHEEL FORKS FOR BUG 4R
B13	302263	PAIR OF FRONT WHEELS FOR BUG 4R
B20	301993N	COMPLETE FRONT FRAME OF THE 869 BASE BUG Small (BLACK)
POS.	PART NR.	DESCRIPTION


Spare Parts FRONT FRAME OF THE 869 BASE size Small



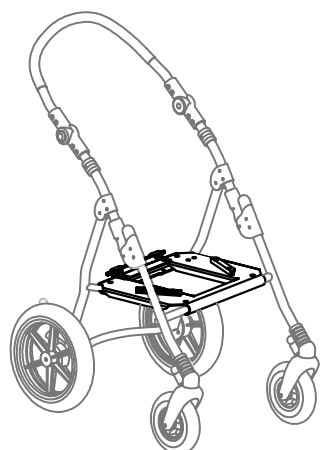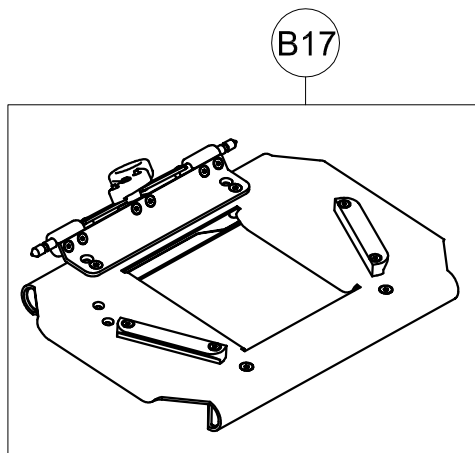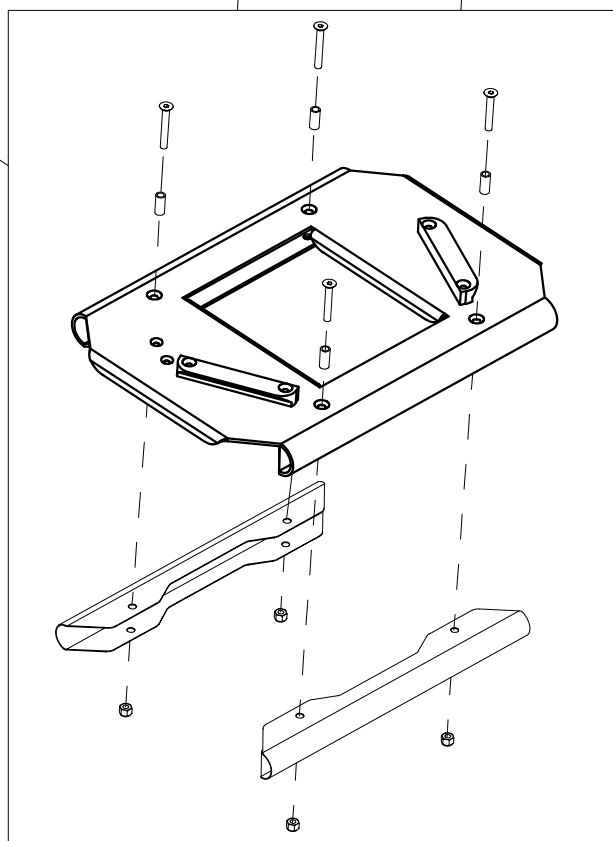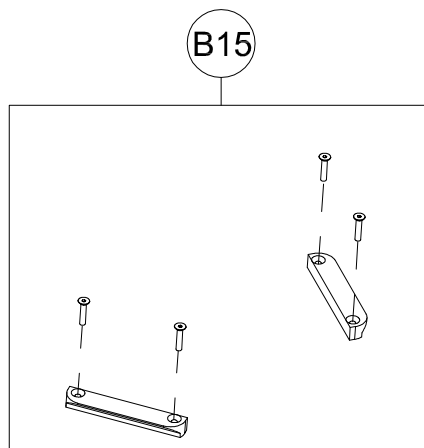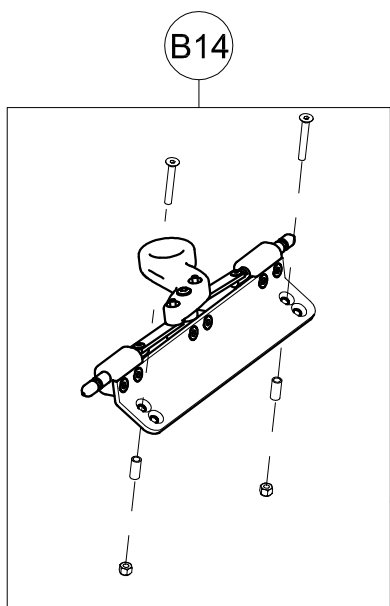

B14	301367	FIXING PART FOR SEAT OF BUG
B15	301995	PAIR OF THE SEAT SLIDES FOR BUG Small
B16	302462	SHEET-METAL OF THE SEAT FOR 869 BUG Small +SLIDES + SCREWS
B17	302468	SEAT OF THE 869 BASE BUG Small
POS.	PART NR.	DESCRIPTION


Spare Parts SEAT OF THE 869 BASE size Small

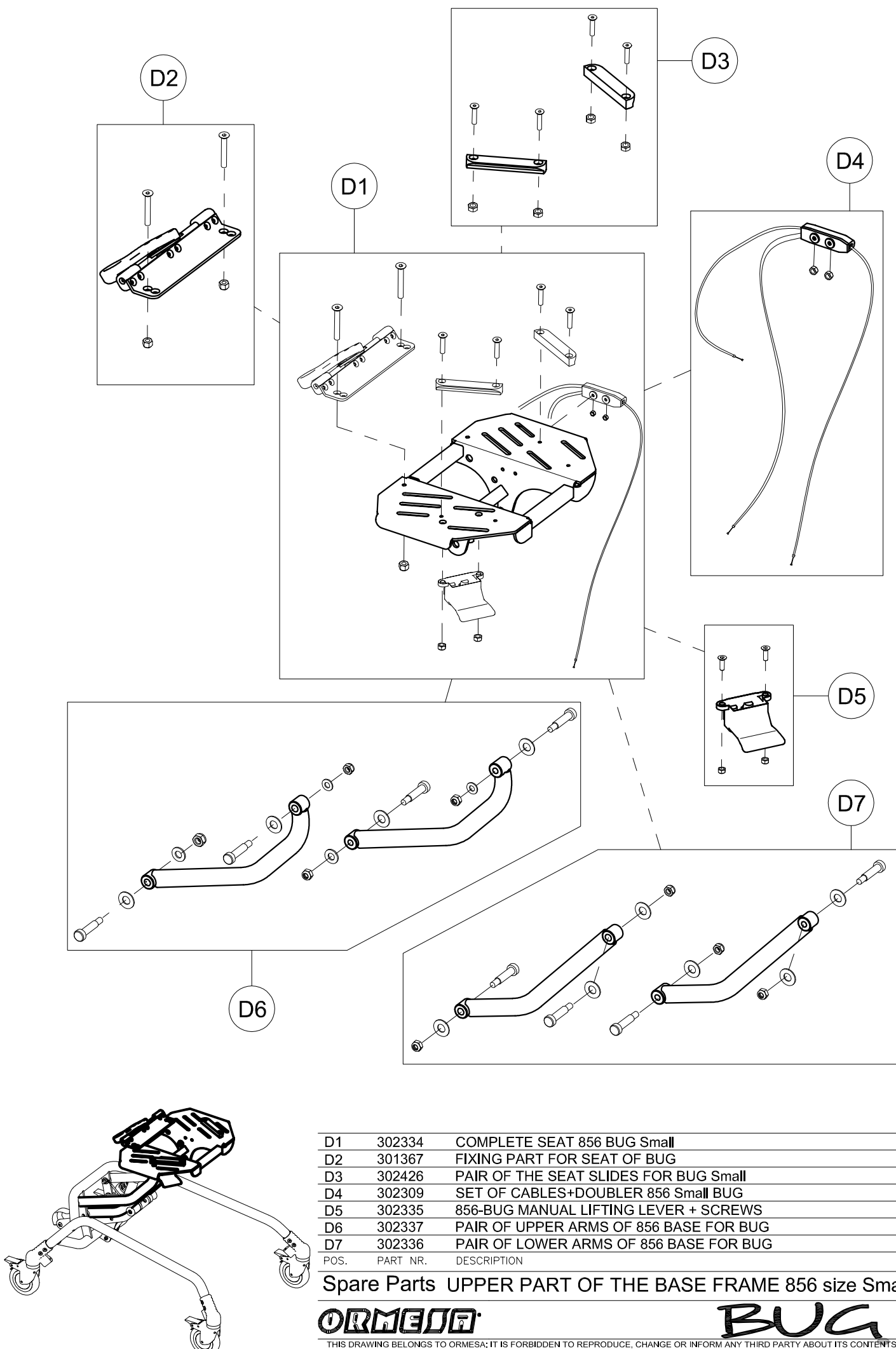

D1	302334	COMPLETE SEAT 856 BUG Small
D2	301367	FIXING PART FOR SEAT OF BUG
D3	302426	PAIR OF THE SEAT SLIDES FOR BUG Small
D4	302309	SET OF CABLES+DOUBLER 856 Small BUG
D5	302335	856-BUG MANUAL LIFTING LEVER + SCREWS
D6	302337	PAIR OF UPPER ARMS OF 856 BASE FOR BUG
D7	302336	PAIR OF LOWER ARMS OF 856 BASE FOR BUG
POS.	PART NR.	DESCRIPTION

**Spare Parts UPPER PART OF THE BASE FRAME 856 size Small**

THIS DRAWING BELONGS TO ORMESA: IT IS FORBIDDEN TO REPRODUCE, CHANGE OR INFORM ANY THIRD PARTY ABOUT ITS CONTENTS

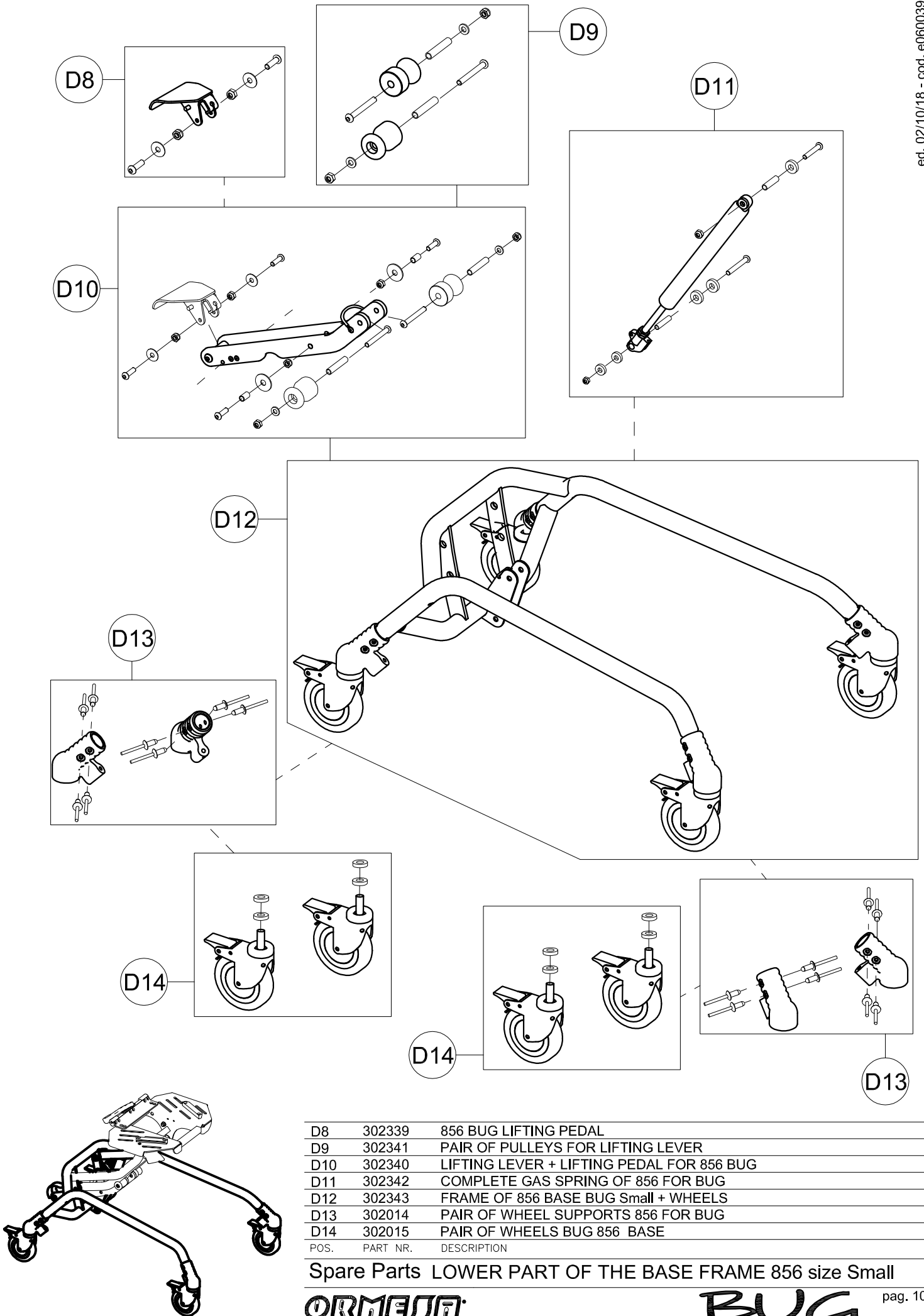

D8	302339	856 BUG LIFTING PEDAL
D9	302341	PAIR OF PULLEYS FOR LIFTING LEVER
D10	302340	LIFTING LEVER + LIFTING PEDAL FOR 856 BUG
D11	302342	COMPLETE GAS SPRING OF 856 FOR BUG
D12	302343	FRAME OF 856 BASE BUG Small + WHEELS
D13	302014	PAIR OF WHEEL SUPPORTS 856 FOR BUG
D14	302015	PAIR OF WHEELS BUG 856 BASE
POS.	PART NR.	DESCRIPTION

Spare Parts LOWER PART OF THE BASE FRAME 856 size Small

**ORMESA**

**BUG**