
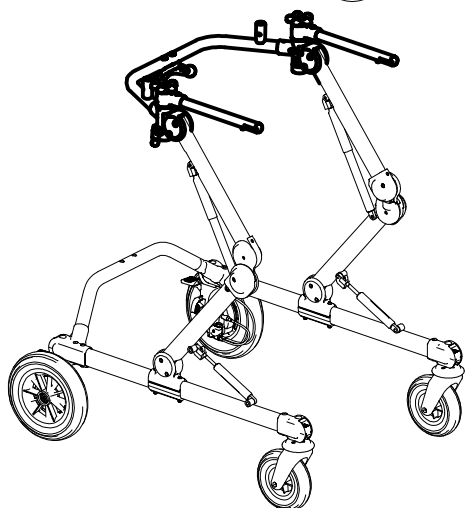





A	103641	925-L TRUNK SUPPORT
B	103637	924-L PELVIC SUPPORT
C	103633	930-L PELVIC HARNESS
D	103613	809-L ARM SUPPORT
G	102810	890-C Large LEG DIVIDER+ABDUCTOR
H	302450	RIGHT-LEFT SUPPORTS FOR 809
J	303247	POSTERIOR FRAME GRILLO Large
M	103588	927 REMOVABLE GRIP HANDLES POSTERIOR GRILLO
N	103656	928 med/large REAR WHEELS NARROW BASE
R	105605	810 M-L PAIR OF WEIGHT BARS
S	102815	890-SC Large LEG DIVIDER
T	102970	939 ERGONOMIC HANDLES
V	103616	943 M-L ARM STRAPS
POS.	PART NR.	DESCRIPTION

Spare Parts POSTERIOR GRILLO size Large

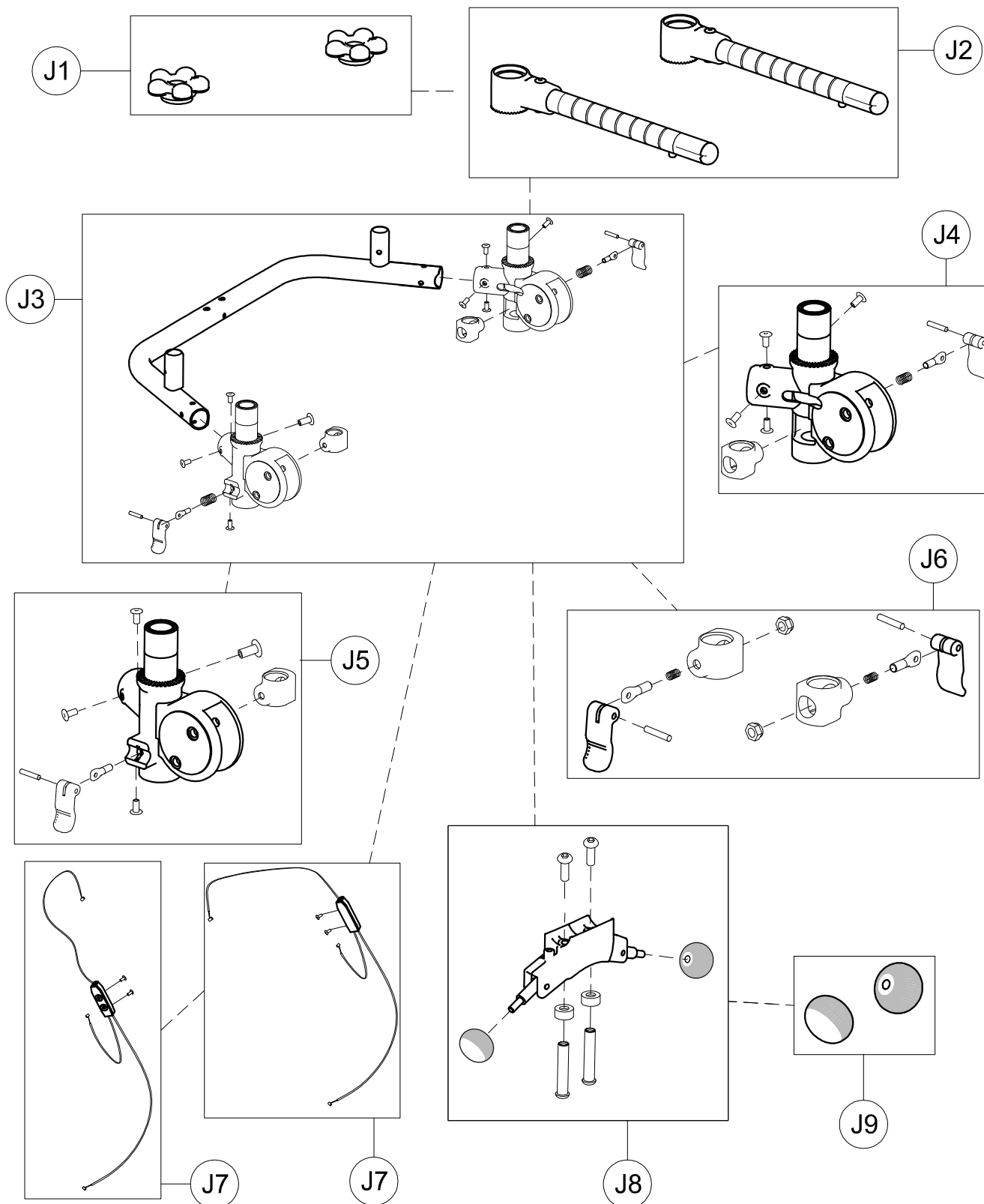


J1	301797	PAIR OF KNOBS TO FIX HORIZONT.SUPPORT.
J2	302284	PAIR OF HORIZ.BARS FOR HANDLEBAR GRILLO large
J3	303236	UPPER CROSSPIECE SUPPORT+LOCKING LEVERS
J4	302286	RIGHT UPPER CROSSPIECE SUPPORT+LOCKING LEVER
J5	302287	LEFT UPPER CROSSPIECE SUPPORT+LOCKING LEVER
J6	302288	SET OF LEVERS TO LOCK TRUNK SUPPORT VERTICALLY
J7	303406	CABLES SET+ DOUBLER
J8	302290	COMMAND SUPPORT + LEVERS AND KNOBS
J9	301808	PAIR OF KNOBS FOR GAS SPRING GRILLO
POS.	PART NR.	DESCRIPTION

Spare Parts UPPER PART OF POSTERIOR GRILLO BASE size Large

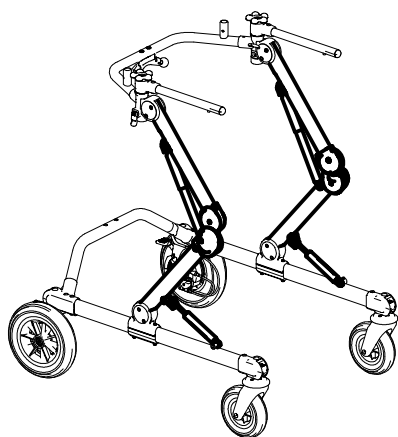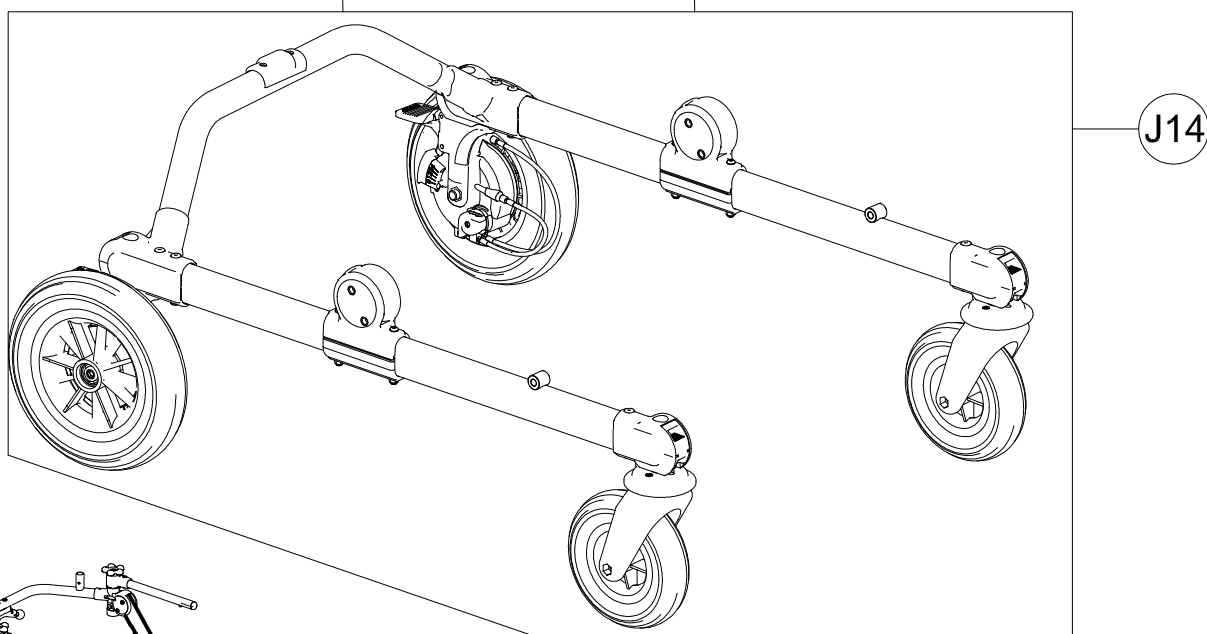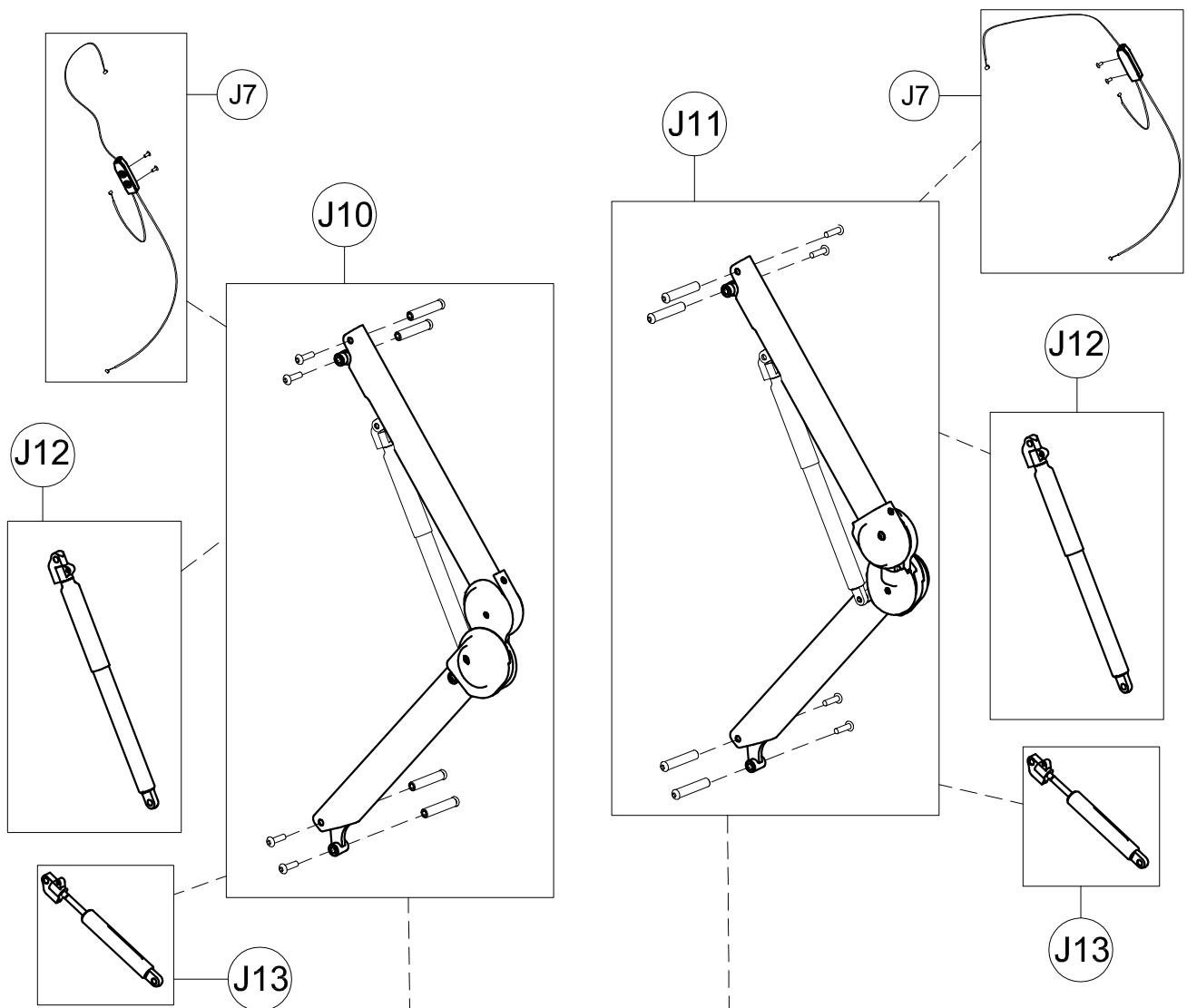


J7	303406	CABLES SET+ DOUBLER
J10	302293	COMPLETE LEFT SIDE
J11	302292	COMPLETE RGT SIDE
J12	302033	UPPER GAS SPRING
J13	302395	LOWER GAS SPRING
J14	303248	POSTERIOR GRILLO Large BASE + WHEELS
POS.	CODICE	DESCRIZIONE

Spare Parts POSTERIOR GRILLO SIDES size Large

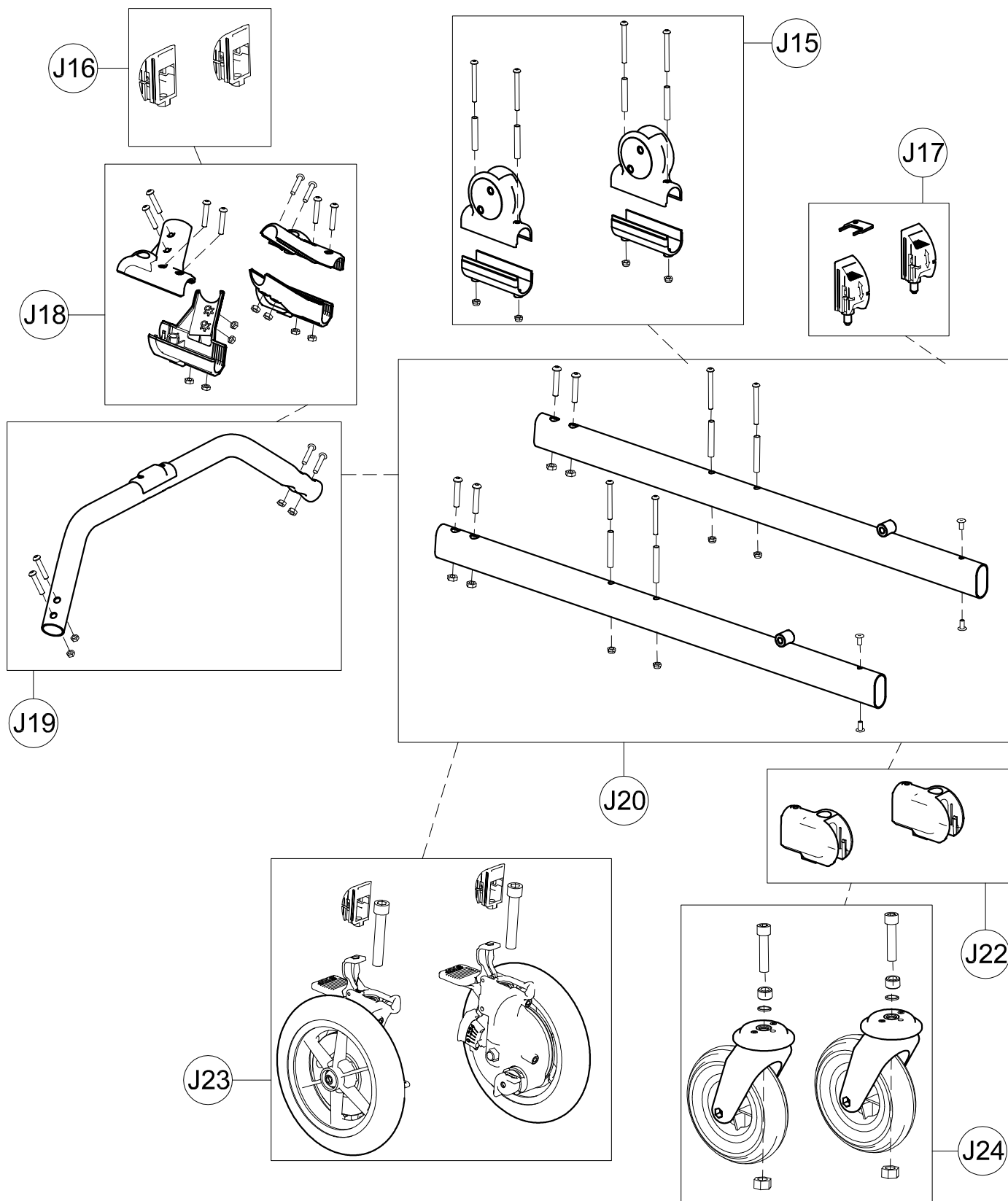

J15	301812	PAIR OF UPPER/LOWER SUPPORTS OF SIDE
J16	303508	PAIR OF DIRECT.CAPS FOR WHEELS
J17	302546	PAIR OF DIRECTION LOCKS
J18	303211	KIT OF LOWER CROSSPIECE SUPPORT
J19	303239	LOWER CROSSPIECE+SCREWS
J20	303238	PAIR OF BASE SIDES
J22	303208	PAIR OF WHEEL SUPPORTS
J23	303476	PAIR OF COMPLETE REAR WHEELS
J24	303387	PAIR OF COMPLETE FRONT WHEELS
POS.	CODICE	DESCRIZIONE

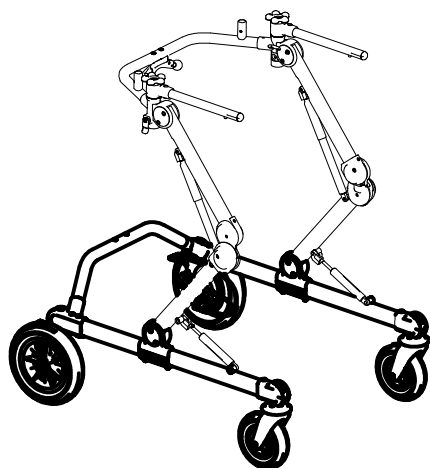

Spare Parts


LOWER PART OF POSTERIOR
GRILLO BASE size Large

pag. 4

ORMESA®

GRILLO



QUESTO DISEGNO È DI PROPRIETÀ ORMESA; È VIETATO RIPRODURNE, MODIFICARNE O COMUNICARNE A TERZI IL CONTENUTO

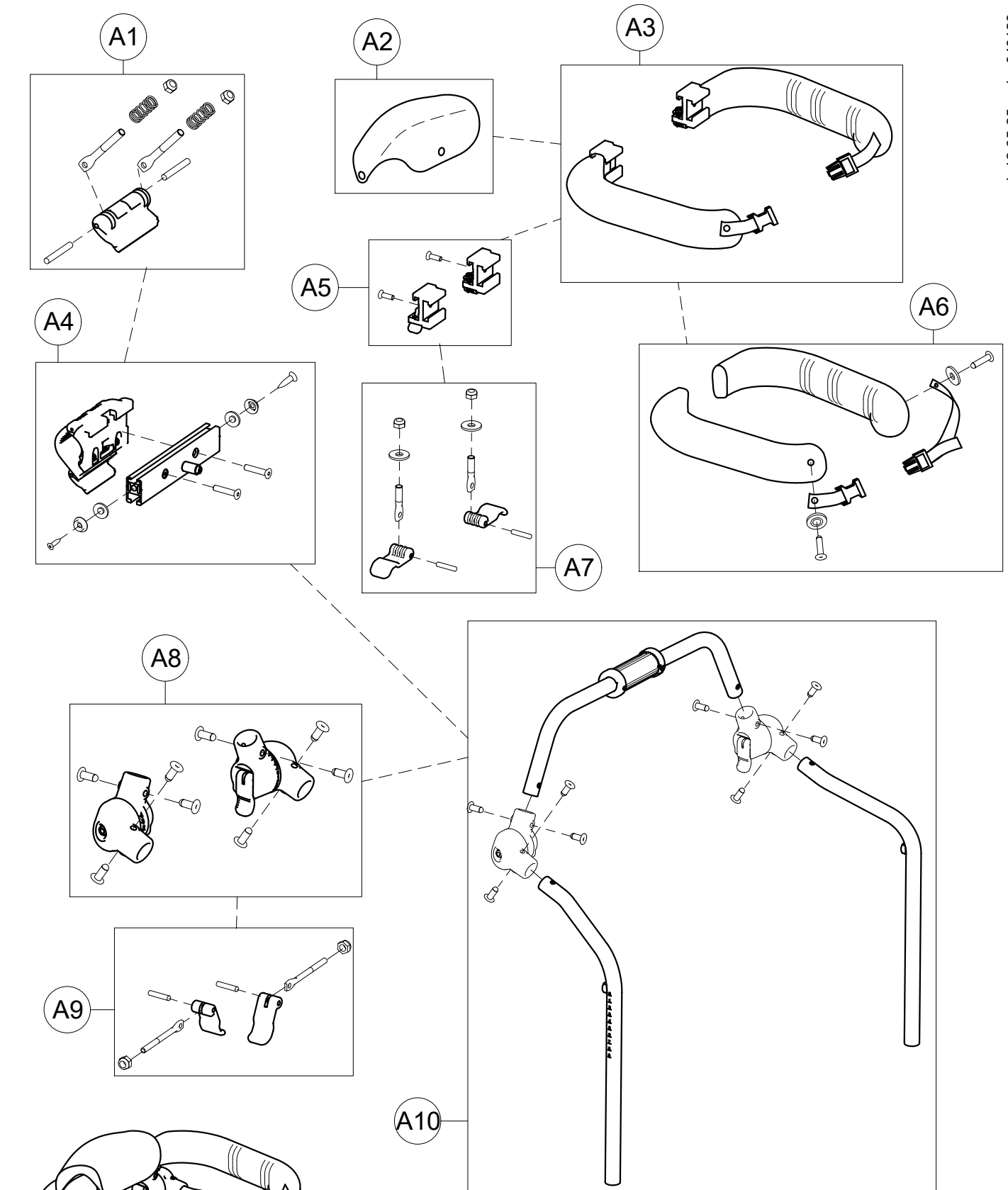

A1	301832	SET OF CLAMPING LEVER TO ROTATE TRUNK SUPPORT
A2	160571P	CENTRAL PADDING
A3	303190	PAIR OF TRUNK HALF SUPPORT
A4	303240	EXTERNAL ROTATION SUPPORT + WIDTH ADJUSTMENT
A5	302298	PAIR OF PROFILE TO SLIDE TRUNK/PELVIC SUPPORT
A6	303241	PAIR OF TRUNK PADDING + STRAPS AND RIVETS
A7	301842	SET OF LEVERS FOR TRUNK SUPPORT WIDTH
A8	303223	PAIR OF TRUNK SUPPORT JOINTS
A9	301843	SET OF TRUNK SUPPORT JOINT LEVERS
A10	303242	FRAME+JOINTS TRUNK SUPPORT



POS. PART NR. DESCRIPTION

Spare Parts TRUNK SUPPORT size Large

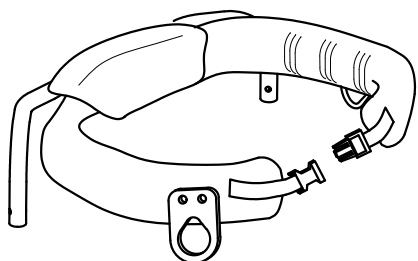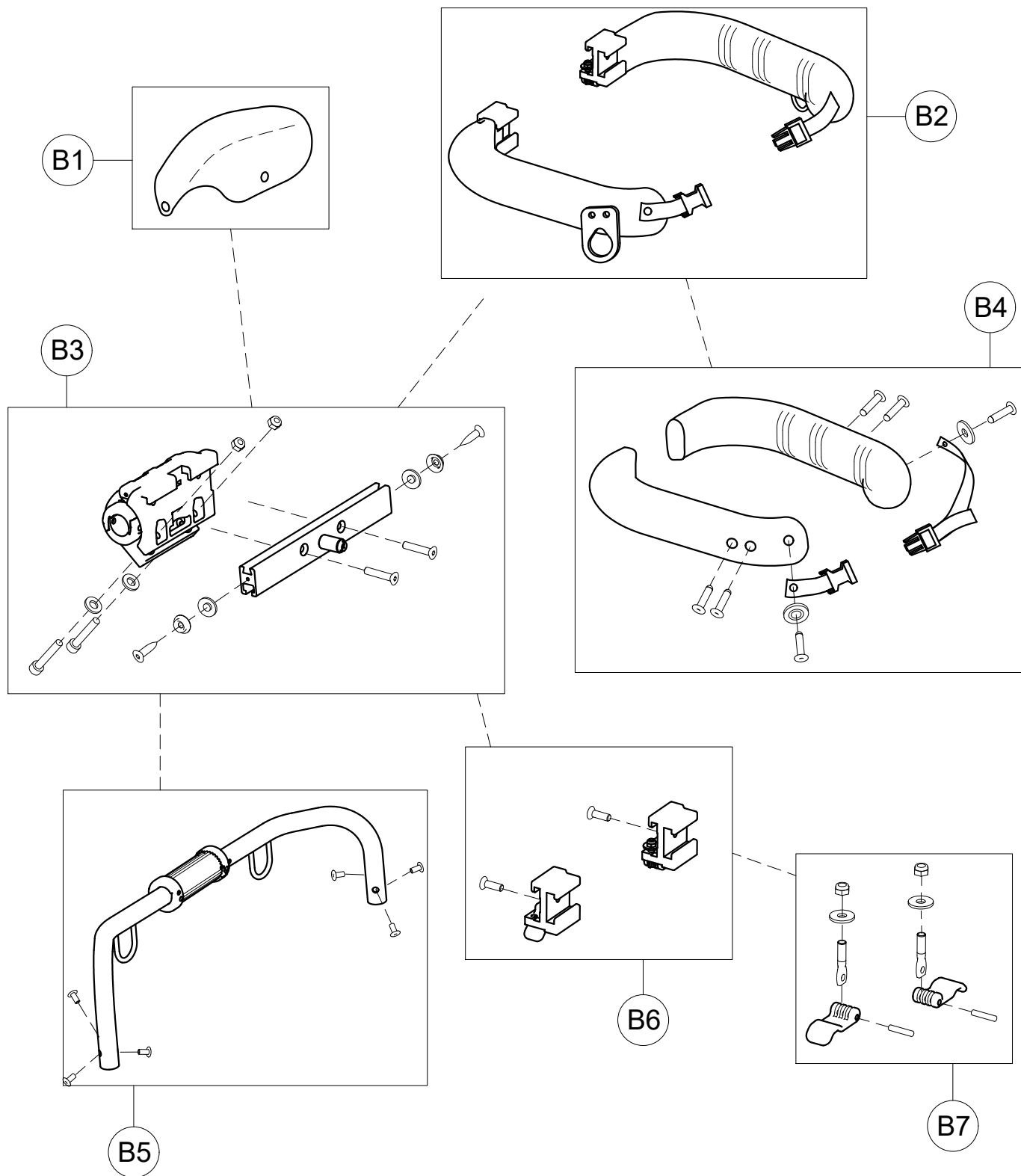

B1	160571P	CENTRAL PADDING
B2	303189	PAIR OF PELVIC HALF SUPPORT
B3	303244	EXTERNAL SUPPORT + PELVIC WIDTH ADJUSTMENT
B4	303245	PAIR OF PELVIC PADDING + STRAPS AND RIVETS
B5	303246	PELVIC TRAVERSE SUPPORT
B6	302298	PAIR OF PROFILE TO SLIDE TRUNK SUPPORT
B7	301842	SET OF LEVERS FOR TRUNK SUPPORT WIDTH
POS.	PART NR.	DESCRIPTION

Spare Parts PELVIC SUPPORT size Large

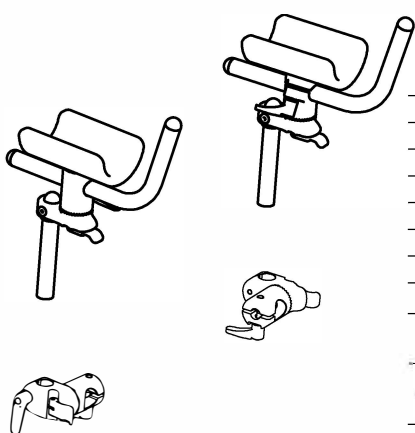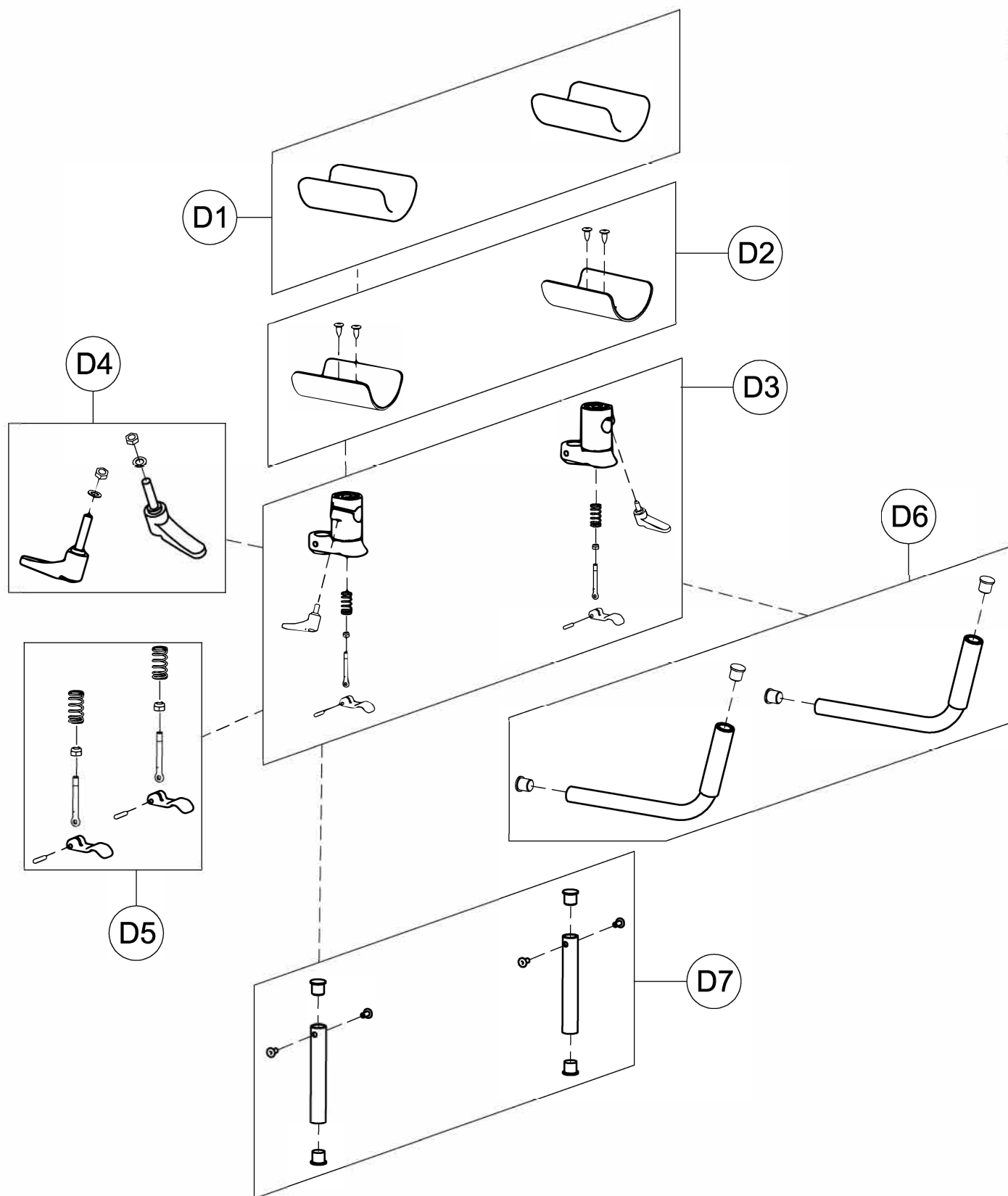

D1	171254	809 medium/large ARM SUPPORT UPHOLSTERY
D2	301850	PAIR OF CURVED PLATE 809 medium/large
D3	301851	PAIR OF LOWER/UPPER TOOTHED SUPPORTS 809
D4	303108	PAIR OF M6X25 LEVERS
D5	301852	SET OF TOOTHED SUPP.LEVERS 809
D6	301853	PAIR OF HANDLEBAR BARS 809
D7	301855	PAIR OF VERT.BARS 809
POS.	CODICE	DESCRIZIONE



Spare Parts ARM SUPPORTS size Medium and Large

pag. 7

ORMESA®



GRILLO

QUESTO DISEGNO È DI PROPRIETÀ ORMESA: È VIETATO RIPRODURNE, MODIFICARNE O COMUNICARNE A TERZI IL CONTENUTO

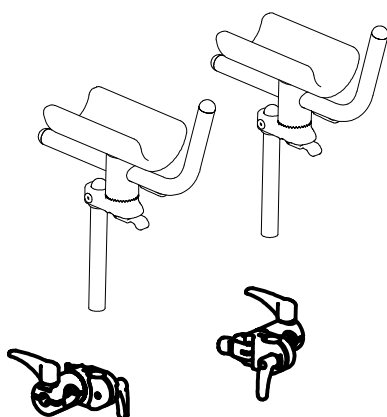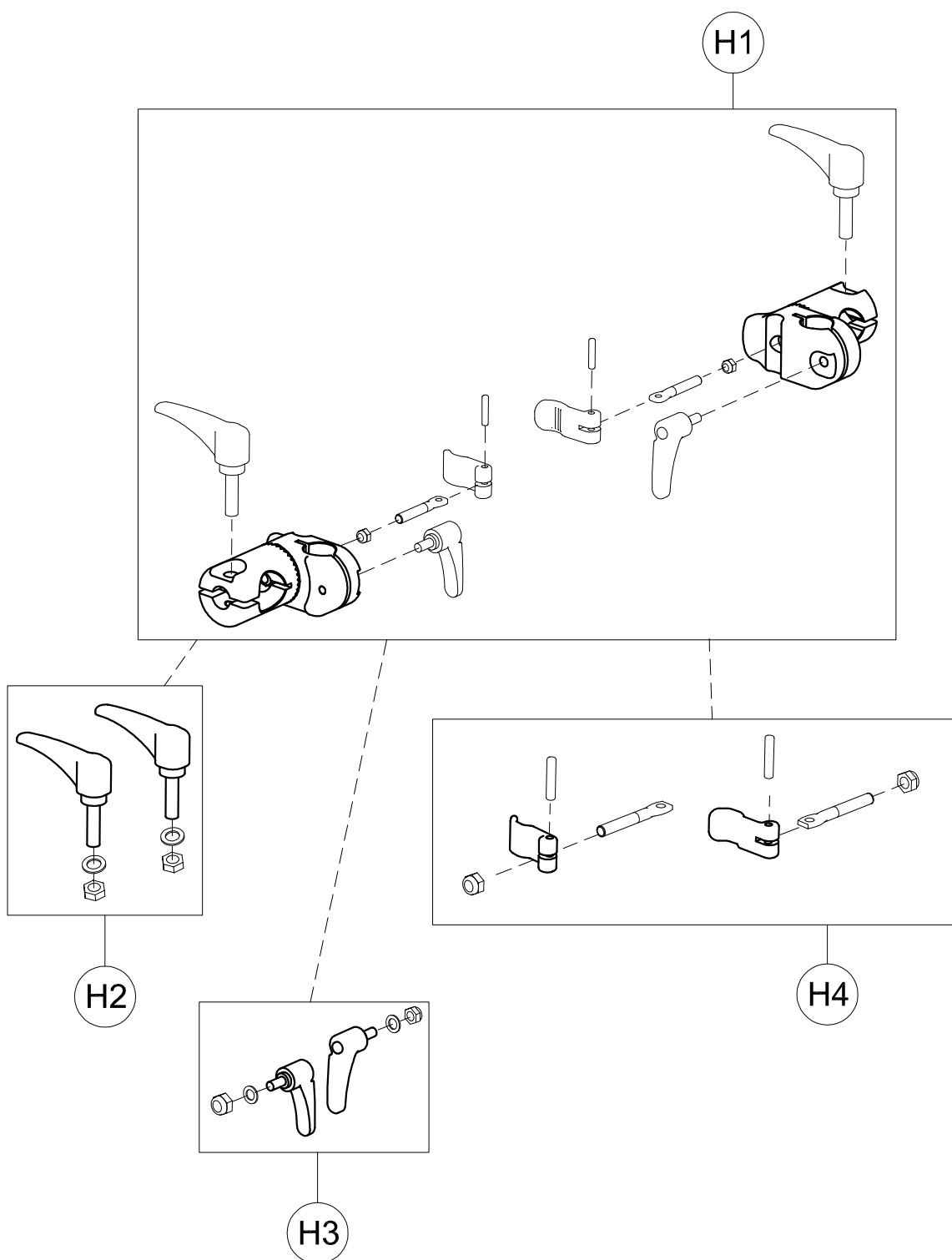

H1	302450	RGT-LEFT SUPPORTS OF 809 GRILLO
H2	301799	PAIR OF M8X35 HANDLES
H3	301800	PAIR OF M6X30 HANDLES
H4	301801	SET OF LEVERS TO LOCK THE HANDLEBAR VERTICALLY
POS.	PART NR.	DESCRIPTION

Spare Parts **ARM SUPPORT JOINT FOR POSTERIOR GRILLO**
size Small, Medium and Large

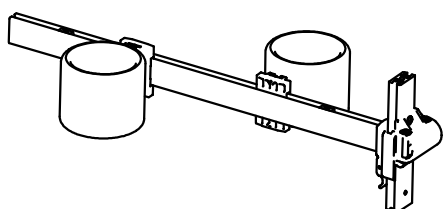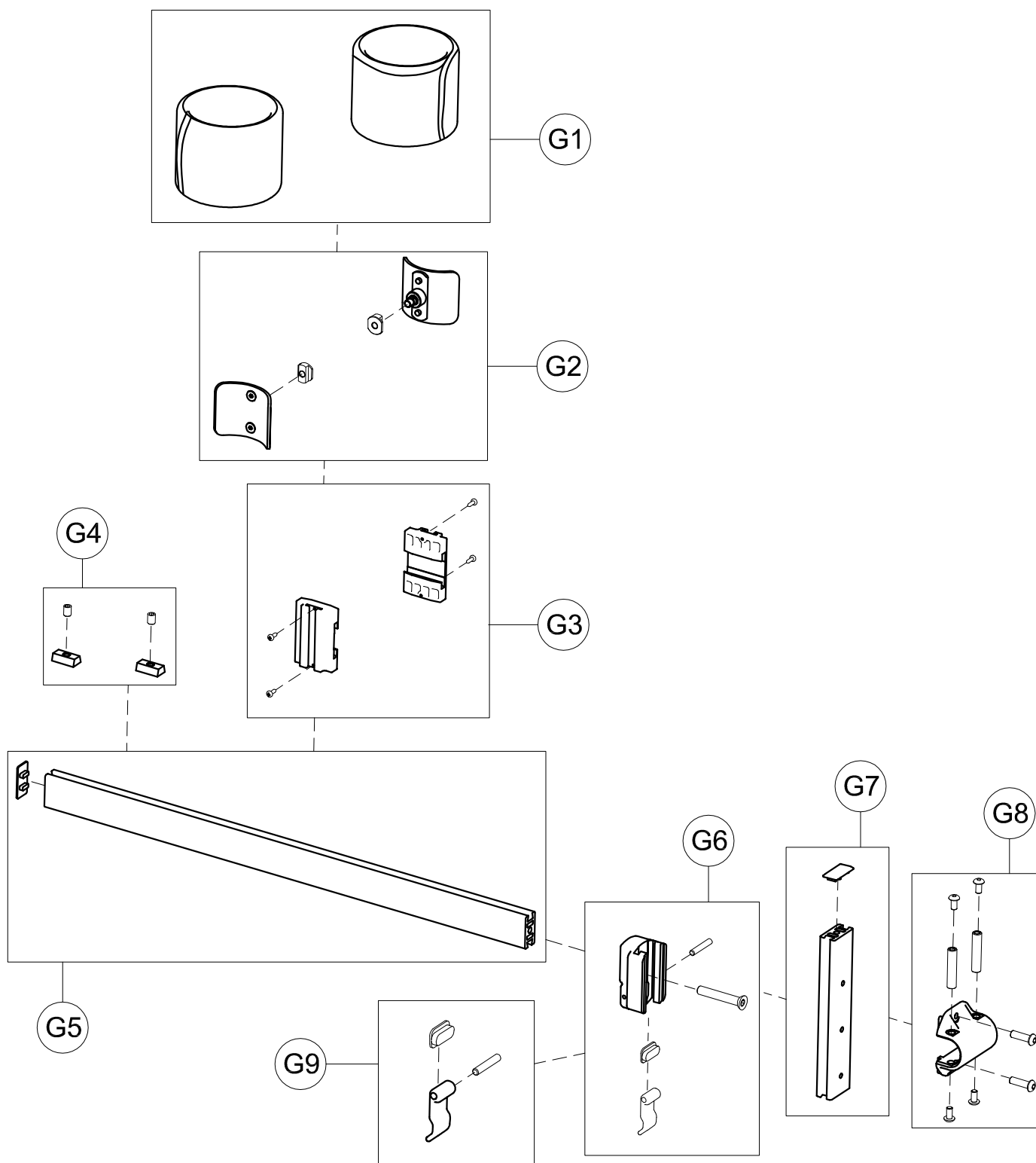

G1	171535	890 large(5) UPHOLSTERY for GRILLO
G2	302933	PAIR OF CURVED PLATE 890 large(5) GRILLO
G3	301858	PAIR OF ABDUCT.SLIDING PROFILE GRILLO
G4	301859	PAIR OF ABDUCT. 890 STOP GRILLO
G5	302304	HORIZONTAL BAR 890 large(5) + CAP GRILLO
G6	301862	890 FIXING BAR +LEVER/SCREWS
G7	303232	890 VERTICAL BAR + CAP
G8	303233	FIXING LOCK 890 + SCREWS
G9	301866	SET OF HEIGHT ADJUST.LEVER 890 GRILLO
POS.	PART NR.	DESCRIPTION

Spare Parts LEG DIVIDER+ABDUCTORS size Large

pag. 9

ORMESA®

GRILLO

THIS DRAWING BELONGS TO ORMESA: IT IS FORBIDDEN TO REPRODUCE, CHANGE OR INFORM ANY THIRD PARTY ABOUT ITS CONTENTS